

Adjustable Platform Bath Step K40

LANGHAM
INSPIRED BY YOUR NEEDS



The Langham Bath Step is unique in its design and versatility, unlike traditional bath steps with one or two fixed heights the Langham Bath Step is modular; allowing extra sections to be added or removed as required.

Each section is 25mm thick (1"), each section has four strong locator pins that slot into the section below, building into a secure, stable platform.

The step is supplied with four sections giving a basic step of 100mm (4"), sections can be removed or extra sections purchased to build the step up. The bottom section has four rubber non-slip feet to provide grip and prevent the step from moving. The surface of each section is slightly textured to give a non-slip surface, without making it uncomfortable on the feet.

The Langham Bath Step is 450mm (18") long by 350mm (14") wide. This provides ample area to position both feet for extra stability whilst climbing into or out of the bath. This is particularly important where stepping over the side is not the main concern, balance is. The step should be set at the same height as the inside base of the bath, this ensures that the feet are at the same height when one foot is in the bath and the other is on the step, helping to reduce the possibility of over balancing.

Adjustable Platform Bath Step K40



As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement, when using this equipment.

1. Ensure the step is dry and free from soap or oil film.
2. Position step up to side of bath.
3. To adjust the height of the step, remove sections by easing apart each corner using a blunt edge tool. Ensure the remaining sections are firmly pushed together.
4. Test for stability.
5. Stand centrally on the step with both feet.
6. Step directly into the bath.
7. Clean with non-scratch detergent.
8. The product is designed for use in a static mode only.

Maximum operating load



500-kg (70 stone)

LANHAM
INSPIRED BY YOUR NEEDS



TRENT LANE
CASTLE DONINGTON
DERBY DE74 2PY
ENGLAND
TEL: (01332) 810504
INT: + 44 1332 810504
FAX: (01332) 850830

enquiries@gordonellis.co.uk
www.gordonellis.com

K40/0216