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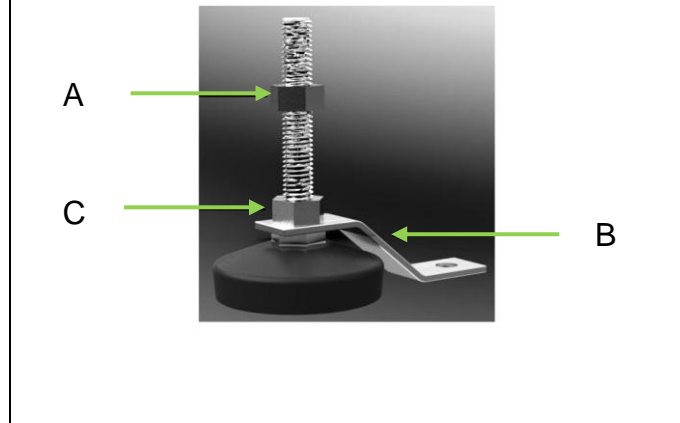
#### English

The Bigfoot Half Step is designed to assist people who are unsteady on their feet to achieve a more secure transfer.

Do not use the step in slippery conditions, for example if covered in ice. If fitting the Security Fixing kit please refer to the installation guide on the other side of the page.

1. Screw the feet in to the Adjustable Legs. Position the Half-Step alongside the building entrance or threshold. The longest side of the step should be pushed up against the entrance or other static surface.
2. If necessary adjust the height of the step by pressing the red button and turning the adjustable leg. Each press of the button will allow one turn of the leg. Fine adjustment to level the step is done by adjusting the feet using the supplied spanner. This will enable a sound and level step to be set.
3. Before use check the step is secure.
4. Tread centrally onto the step surface avoiding stepping onto the corners or along the edges of the step, do not lean backwards as this is less stable for the user.
5. The Half-Step and the ground the step is placed on must be kept clean to avoid slippage. The step can be cleaned using any all-purpose, household cleaner not containing bleach.

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