**Gordon Ellis & Co**

Trent Lane  
Castle Donington  
Derby DE74 2AT  
England

Tel: (01332) 810 504  
Fax: (01332) 850 830  
enquiries@gordonellis.co.uk  
www.gordonellis.com

Company registered in England  
No. 231734

VAT No. GB 428 4666 26

**Part number:**  
6007, 6007/H

**Max Weight Limit:**  
30 stone, 190kg, 420lbs

**Made in UK**  
Registered Design

## Derby Bath Board

The Derby Bath Board fits over the bath to help those with stiff joints or poor balance to have a safe and relaxing shower or bath. Please do not use for any other purpose.

### Box Includes:

- 1 x Derby Bath Board
- 4 x Pillars
- 4 x Screws
- 1 x Handle (only with product code 6007/H)

### Fitting Instructions:

1. Fasten two pillars at each end of the bath board through the outermost slots with two screws. If fitting the handle to the board, insert the screws into the holes in the handle before putting through the board and screwing into the pillars (see diagram 1).
2. Position the bath board over the bath ensuring that the rear upstand is on top.
3. Adjust the pillars for a secure fit against the sides of the bath (see diagram, 2). Avoid the board overhanging the side of the bath.
4. Tighten the wingnuts to secure the pillars in positions (see diagram 3).
5. Test the board to ensure it is stable on the bath.
6. Check the security of the board before each use.
7. Clean with a non-abrasive disinfectant.

### Please Note:

- Individuals who use a bath board need to be able to lift their legs over the bath side, or have assistance to do so from a carer.
- Keep away from children unless supervised.
- Some users may find it appropriate to place a folded towel on the board to prevent risk of trapping parts of the body.

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