**EN** PROVIDES SUPPORT AND STABILITY BY ACTING AS BOTH A RAISED TOILET SEAT AND A TOILET SURROUND RAIL. BUILT IN ARMS SUPPORT THE USER IN A HIGH POSITION FOR EASIER TRANSFER, WHILE THE 10CM (4IN) RISE MAKES IT EASIER TO STAND FROM A SEATED POSITION.

IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK FOR STABILITY BEFORE EACH USE. PERIODICALLY CHECK FIXINGS FOR TIGHTNESS.

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# ENTERPRISE

## RAISED TOILET SEAT WITH ARMS

Welcome to your new Gordon Ellis product.  
 We hope that it proves a useful aid in your daily life.

**SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS**  
 AVAILABLE IN THE FOLLOWING LANGUAGES:  
 EN FR DE IT ES


MAXIMUM USER WEIGHT	POIDS MAXIMUM DE L'UTILISATEUR
MAXIMALES BENUTZERGEWICHT	PESO MASSIMO SOPPORTATO
190 Kg	PESO MÁXIMO DEL USUARIO
420 lbs	

**EN** ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW  
**FR** TOUT ACCIDENT GRAVE SURVENANT PENDANT L'UTILISATION DOIT ÊTRE SIGNALÉ AU EC REP - COORDONNÉES CI-DESSOUS  
**DE** JEDER SCHWERE UNFALL WÄHREND DES GEBRAUCHS MUSS EC REP GEMELDET WERDEN - KONTAKTDATEN UNTEN  
**IT** QUALSIASI INCIDENTE GRAVE CHE SI VERIFICA DURANTE L'USO DEVE ESSERE SEGNALATO AL PRODUTTORE - DETTAGLI DI CONTATTO DI SEGUITO  
**ES** CUALQUIER ACCIDENTE GRAVE QUE OCURRA DURANTE EL USO DEBE INFORMARSE A EC REP - DETALLES DE CONTACTO A CONTINUACIÓN

Your ENTERPRISE RAISED TOILET SEAT WITH ARMS was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.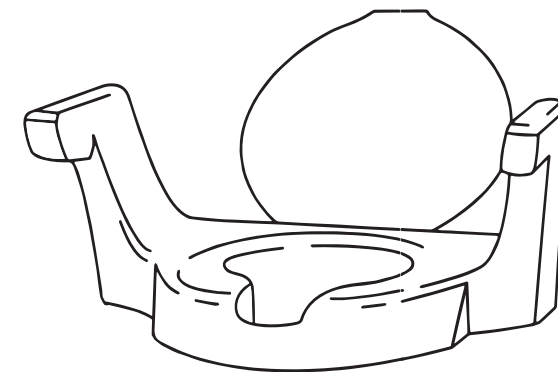

**CLEANING INSTRUCTIONS** INSTRUCTIONS DE NETTOYAGE  
**REINIGUNGSANWEISUNGEN** ISTRUZIONI PER LA PULIZIA  
**INSTRUCCIONES DE LIMPIEZA**

**EN** WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.  
**FR** ESSUYEZ AVEC UN CHIFFON NON ABRASIF ET DE L'EAU SAVONNEUSE.  
**DE** MIT EINEM NICHT SCHEUERNDEN TUCH UND SEIFENWASSER ABWISCHEN.  
**IT** PULIRE CON UN PANNINO NON ABRASIVO E ACQUA SAPONOSA.  
**ES** LIMPIELO CON UN PAÑO NO ABRASIVO Y AGUA CON JABÓN.

PH 5 - 9

DESIGNED & MADE IN THE UK

MD

IFU-REF 6241/1/1120-1

**GORDON ELLIS & CO.**  
 TRENT LANE  
 CASTLE DONINGTON  
 DERBY, ENGLAND DE74 2AT

PART REF 6241/1







EC REP

**AUTHORISED REPRESENTATIVE**  
 ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

## FIXING INSTRUCTIONS
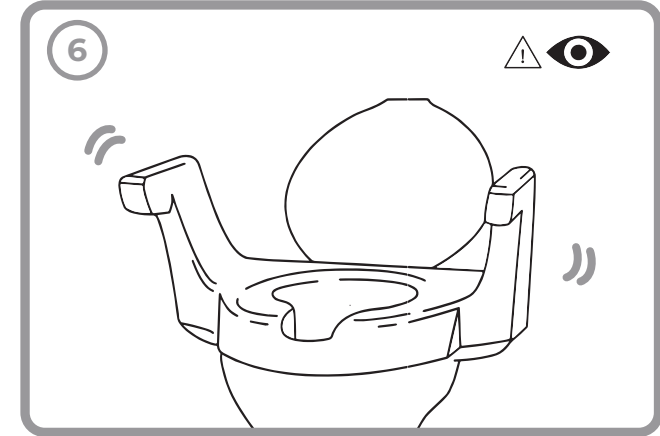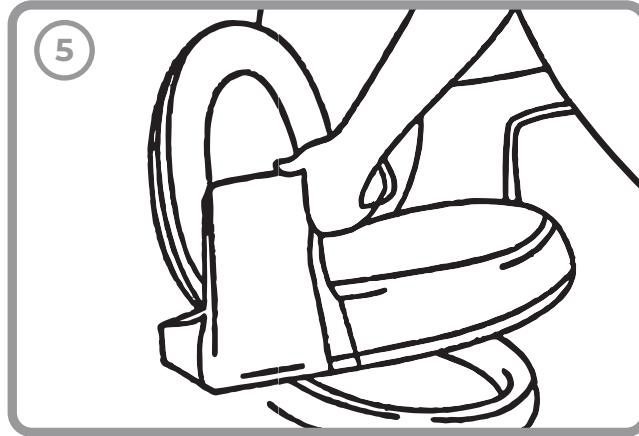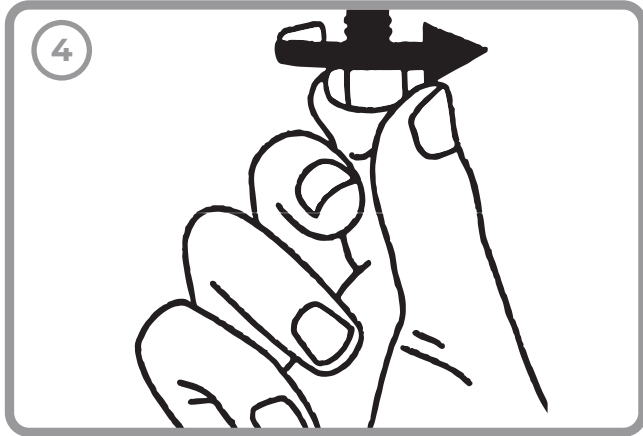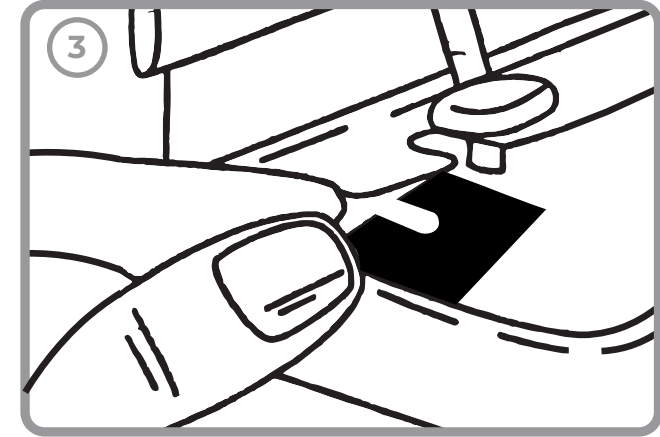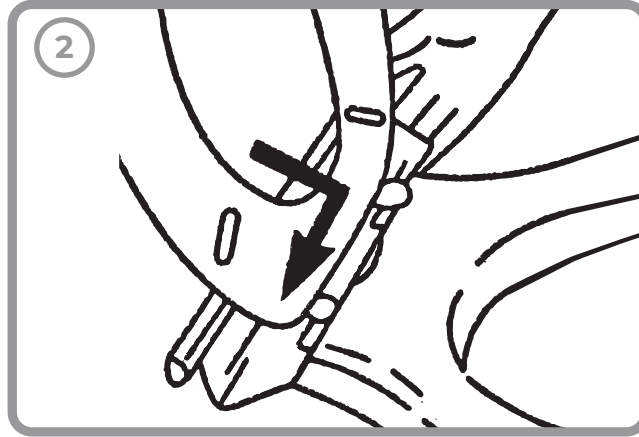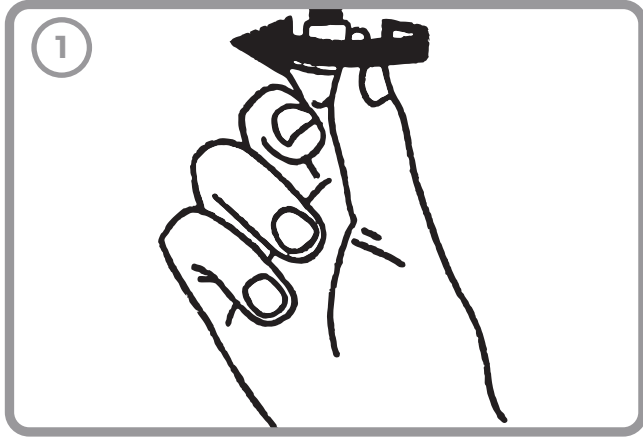
### EN

1. LOOSEN EXISTING TOILET SEAT FROM UNDERNEATH. **DO NOT REMOVE.**
2. SLIDE FITTING BRACKET UNDER EXISTING TOILET SEAT FITTINGS FROM BEHIND.
3. STABILISE WITH ONE OR TWO SLOTTED WASHERS PROVIDED, IF NECESSARY.
4. RE-TIGHTEN TOILET SEAT FIXINGS.
5. FIT ENTERPRISE BY HOOKING THE BRACKET INTO THE MOST SUITABLE SLOT FOR YOUR TOILET. ENSURE SEAT IS PARALLEL.
6. CHECK STABILITY AND TIGHTNESS BEFORE USE.

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