



LANGHAM[®] K40

ADJUSTABLE PLATFORM BATH STEP

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.



EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS OVER LEAF




SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN



MAXIMUM USER WEIGHT

Your LANGHAM K40 STEP was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

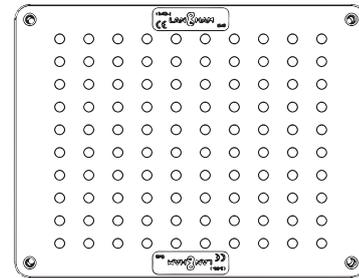

LANGHAM

INSPIRED BY YOUR NEEDS



GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT



AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013
MALTA



MD

IFU-REF LP/K40/1020-2

PART REF LP/K40

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

gordonellis.healthcare
 @EllisHealthcare











INSTRUCTIONS FOR USE

i **EN** UNIQUE AND VERSATILE, UNLIKE TRADITIONAL BATH STEPS WITH ONE OR TWO FIXED HEIGHTS, THE LANGHAM K40 BATH STEP IS MODULAR ; ALLOWING EXTRA SECTIONS TO BE ADDED OR REMOVED AS REQUIRED. SUPPLIED WITH 4 SECTIONS TO GIVE A BASIC STEP OF 100MM (4"), EXTRA SECTIONS CAN BE PURCHASED TO FURTHER BUILD THE STEP UP. EACH SECTION IS SLIGHTLY TEXTURED TO GIVE A NON-SLIP SURFACE WITHOUT BEING UNCOMFORTABLE ON THE FEET.

1. ENSURE THE STEP IS DRY AND FREE FROM SOAP OR OIL FILM.
2. ENSURE THE RUBBER FEET ARE IN PLACE ON THE BOTTOM STEP SECTION.
3. POSITION THE STEP UP TO THE SIDE OF THE BATH IN THE ORIENTATION SHOWN.
4. THE HEIGHT OF THE STEP SHOULD BE THE SAME AS THE INSIDE BASE OF THE BATH IN ORDER TO REDUCE OVER BALANCING.
 - 4.1 ADDING SECTIONS - INSERT THE LOCATOR PINS INTO THE CORRESPONDING HOLES. ENSURE ALL SECTIONS ARE FIRMLY TOGETHER BEFORE USE.
 - 4.2 REMOVING SECTIONS - EASE APART EACH CORNER USING A BLUNT EDGE TOOL. ENSURE ANY REMAINING SECTIONS ARE FIRMLY TOGETHER BEFORE USE.
5. TEST FOR STABILITY.
6. STAND CENTRALLY ONTO THE STEP WITH BOTH FEET BEFORE STEPPING INTO THE BATH.

! DEVICE IS FOR STATIC USE ONLY. IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. IF HEIGHT IS ADJUSTED, CHECK FOR STABILITY BEFORE USE. CHILD AND TODDLER USE UNDER SUPERVISION ONLY, NEVER LEAVE ALONE . DO NOT USE WITH OTHER STEPS.

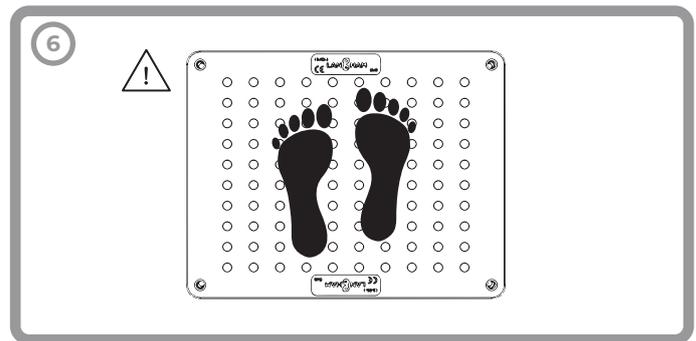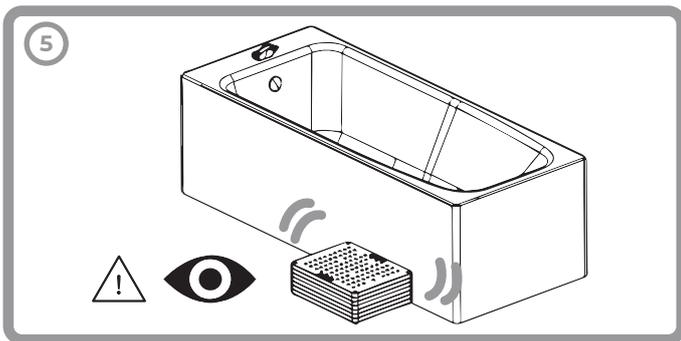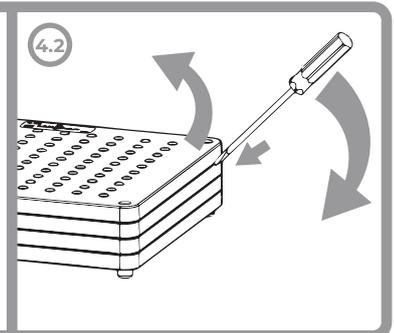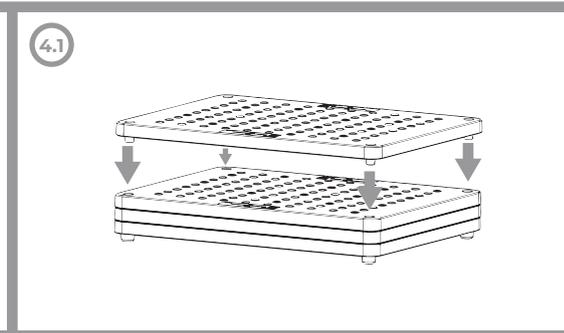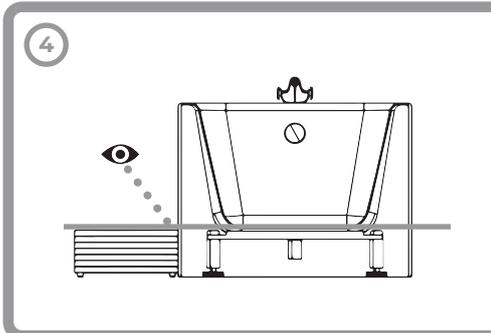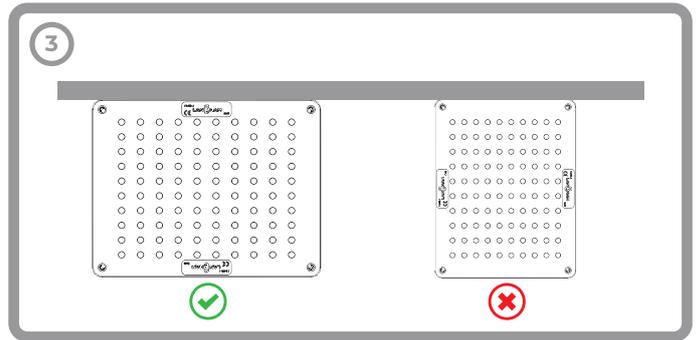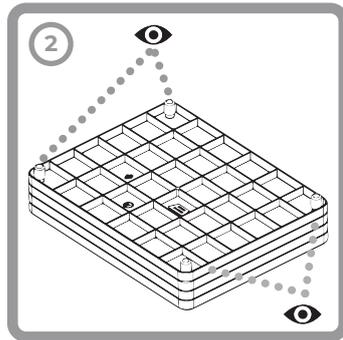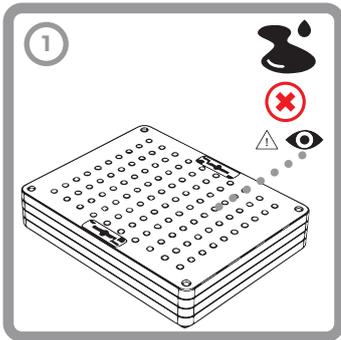

i



PH 5 - 9

CLEANING INSTRUCTIONS

WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)
 @EllisHealthcare

