



SUITABLE FOR:

DERBY HALF STEP 66091 + 66092

LANGHAM HALF STEP LP/PHS



GORDON

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS

AVAILABLE IN THE FOLLOWING LANGUAGES: EN



ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER











| IFU-REF | 61095/0121-1

PART REF | 61095



KIT CONTENTS:

TWO METAL FIXING BARS TWO NUTS




PLEASE READ THESE INSTRUCTIONS ALONG WITH THE INSTRUCTIONS INCLUDED WITH THE HALF STEP.

ENSURE THE STEP IS STABLE BEFORE USE.

PERIODICALLY CHECK AND **RE-TIGHTEN ALL NUTS AND OTHER FIXINGS AS REQUIRED.**

WHEN FITTED, THE DISTANCE FROM THE UNDERSIDE OF THE STEP TO THE END OF THE FOOT MUST BE NO **MORE THAN 3.8CM (6CM FOR 66092)**

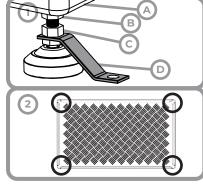


1. FIXING INSTRUCTIONS:

REMOVE FOOT FROM STEP (A). UNSCREW THE LOCKING NUT (B) PLACE THE ANGLED BAR (D) ONTO THE FOOT, THEN SCREW THE NUT (C) TIGHTLY AGAINST IT.

SCREW ON THE LOCKING NUT (B). ENSURE THERE IS THREAD ABOVE THE LOCKING NUT. SCREW FOOT BACK INTO THE STEP (A).

2. IT IS ADVISED THAT THE HALF STEP IS FIXED TO THE FLOOR AT EACH CORNER.

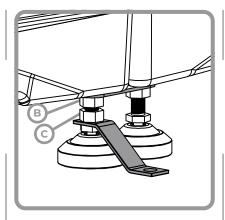

IF USING THE LINKING KIT AND **FIXING KIT TOGETHER:**

THE LINKING KIT GOES ABOVE LOCKING NUT (B)

THE FIXING KIT GOES BELOW BOTH NUTS (B+C)





GORDONELLISHEALTHCARE.CO.UK f gordonellis. healthcare 🍏 @ Ellis Healthcare





SUITABLE FOR:

DERBY HALF STEP 66091 + 66092 LANGHAM HALF STEP

LP/PHS



SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS

AVAILABLE IN THE FOLLOWING LANGUAGES: EN



ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER













KIT CONTENTS:

TWO METAL FIXING BARS TWO NUTS




PLEASE READ THESE INSTRUCTIONS ALONG WITH THE INSTRUCTIONS INCLUDED WITH THE HALF STEP.

ENSURE THE STEP IS STABLE BEFORE USE.

PERIODICALLY CHECK AND **RE-TIGHTEN ALL NUTS AND OTHER** FIXINGS AS REQUIRED.

WHEN FITTED, THE DISTANCE FROM THE UNDERSIDE OF THE STEP TO THE END OF THE FOOT MUST BE NO MORE THAN 3.8CM (6CM FOR 66092)

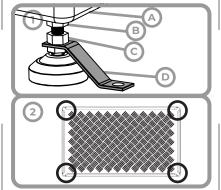


1. FIXING INSTRUCTIONS:

REMOVE FOOT FROM STEP (A). UNSCREW THE LOCKING NUT (B). PLACE THE ANGLED BAR (D) ONTO THE FOOT, THEN SCREW THE NUT (C) TIGHTLY AGAINST IT.

SCREW ON THE LOCKING NUT (B). ENSURE THERE IS THREAD ABOVE THE LOCKING NUT. SCREW FOOT BACK INTO THE STEP (A).

2. IT IS ADVISED THAT THE HALF STEP IS FIXED TO THE FLOOR AT EACH CORNER.

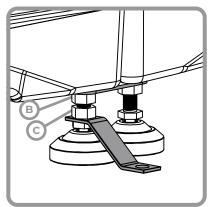

IF USING THE LINKING KIT AND **FIXING KIT TOGETHER:**

THE LINKING KIT GOES ABOVE LOCKING NUT (B)

THE FIXING KIT GOES BELOW BOTH NUTS (B+C)





GORDONELLISHEALTHCARE.CO.UK

| IFU-REF | 61095/0121-1

PART REF | 61095