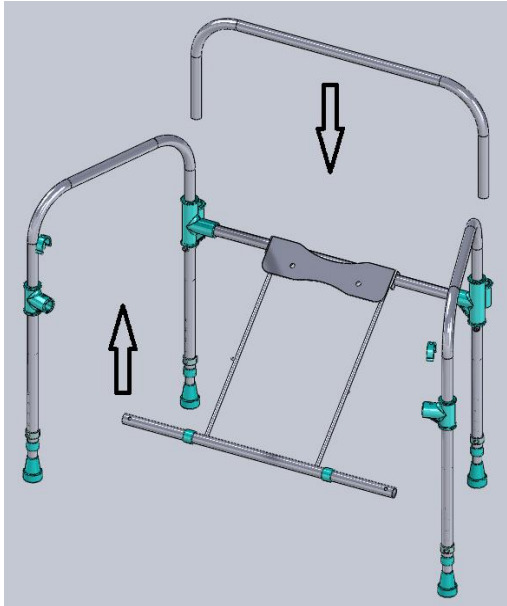
ESPAÑOL

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Big John Commode

ENGLISH

1. Insert the hinges through the left and right side toilet seat and lid.
2. Locate the rubber washers over the holes on the support bracket on the rear crossbar.
3. Mount the toilet seat by locating the two hinges through the holes in the bracket.
4. Screw the wing-nuts on to the bolts making sure they are tight.
5. Unfold the frame and lay on its side on the floor.
6. Remove the two clips and fit the front crossbar to the side frames. Secure by replacing the clips.
7. To adjust height remove the clips from each leg and extend to the required height. Replace the clips.
8. Fit the rear back support by depressing the two pins and pushing in to the rear brackets.
9. May be used over a toilet or fit the pan to the frame to use as a free standing commode.

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ENGLISH

- 1. Test for stability before use.
- 2. The product is designed for use in a static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement during use.
- 3. Clean with mild, household cleaning products. Do not use abrasive powders or scouring pads.

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