
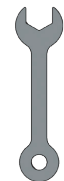
SECURITY FIXING KIT

Tools Required:

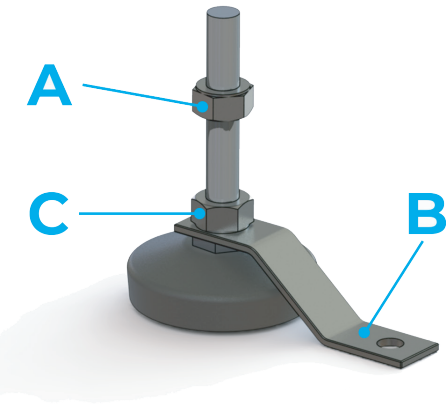10 mm spanner (supplied*)
*in Security Fixing Kit

1. Fit the angled bar (B) onto the foot, screw the nut (C) tightly against it.

2. Refit the feet to the step ensuring the nut (A) is tight against the Step, or base of the Extending Leg (if using).

Please read these instructions, along with the instructions on the reverse.

EN THE ANGLED BARS (B) SHOULD BE FITTED TO DIAGONALLY OPPOSITE CORNERS. AT LEAST 15 MM OF THREAD MUST REMAIN INSIDE THE ADJUSTING LEG FOR SAFE USE.

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PH 5 - 9

EN WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

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VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 gordonellis.healthcare
 @EllisHealthcare

BIG FOOT® HALF STEP HANDLE

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN FR DE ES NL PT DK EL

HANDLES FOR SUPPORT ONLY.
GRIFFE NUR ZUR UNTERSTÜTZUNG.
HANDGREPEN ALLEEN TER ONDERSTEUNING.
KUN TIL STØTTE.

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EN ONLY TO BE USED WITH THE BIG FOOT HALF STEP (66111) - PLEASE READ THESE INSTRUCTIONS ALONG WITH THE STEP INSTRUCTIONS BEFORE USE.

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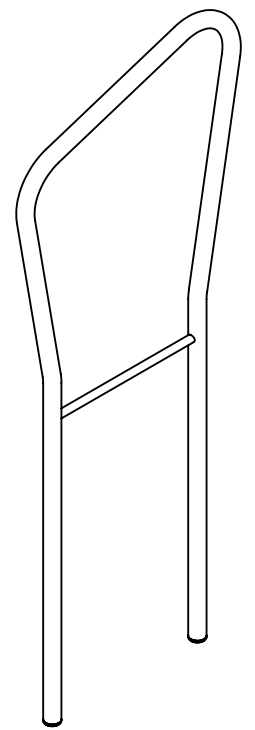
Your BIG FOOT HALF STEP HANDLE was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

 DESIGNED IN THE UK

MADE IN CHINA

PART REF 66115
IFU-REF 66115/1020-3

 GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

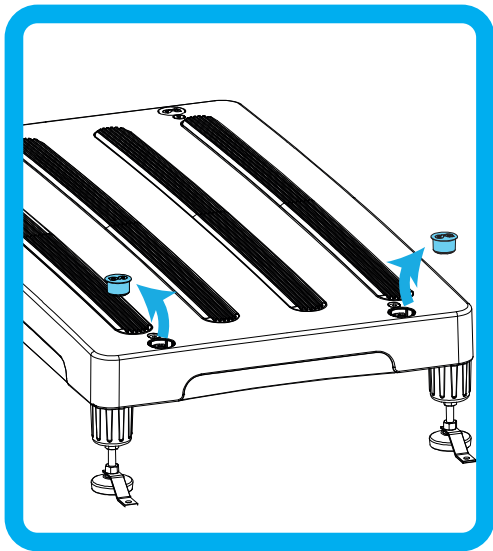
 gordonellis.healthcare
 @EllisHealthcare

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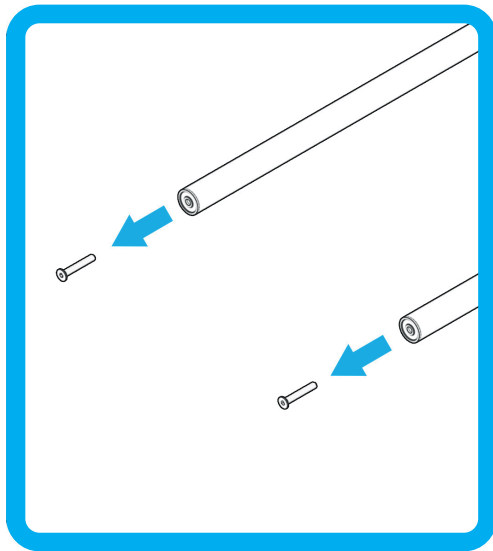
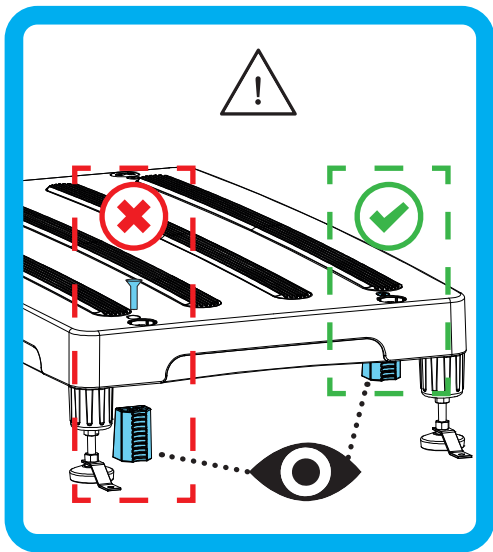
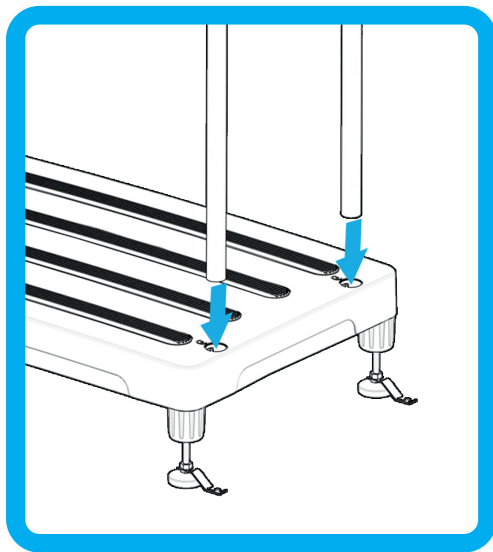
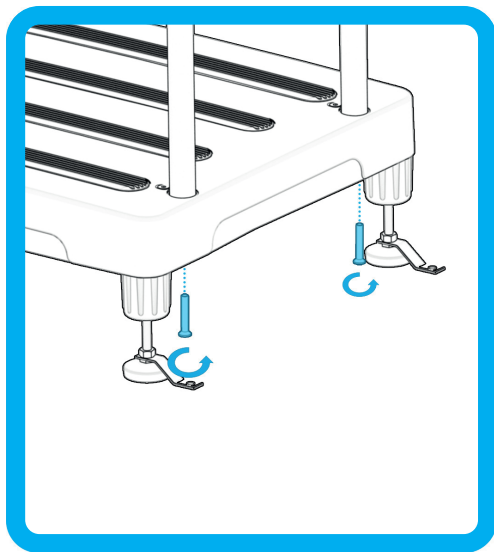
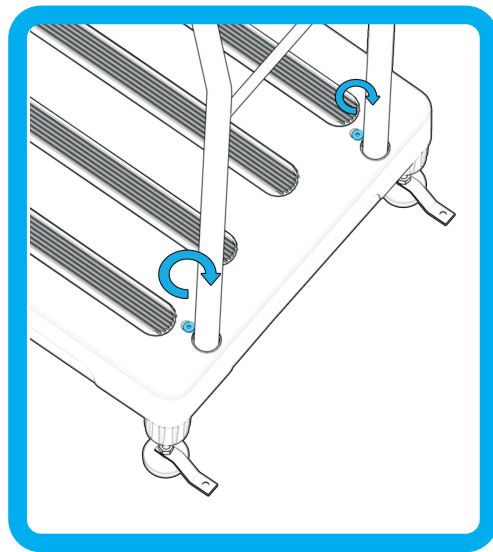
EN THE SUPPORT HANDLE IS DESIGNED TO ASSIST PEOPLE WHO ARE UNSTEADY ON THEIR FEET TO ACHIEVE A MORE SECURE TRANSFER USING THE STEP. IF REQUIRED, A SUPPORT HANDLE CAN BE FITTED TO BOTH SIDES OF THE STEP. IT IS TO BE USED ONLY WHEN FITTED TO A BIGFOOT HALF STEP. INSTALL THE SECURITY FIXING KIT; DO NOT USE THE SUPPORT HANDLE WITHOUT INSTALLING THE SECURITY FIXING KIT. PLEASE REFER TO THE INSTALLATION GUIDE SHOWN ASIDE. BEFORE USE CHECK THE STEP IS SECURE. TREAD CENTRALLY ONTO THE STEP SURFACE AVOIDING STEPPING ON TO THE CORNERS OR ALONG THE EDGES OF THE STEP, DO NOT LEAN BACKWARDS AS THIS IS LESS STABLE FOR THE USER. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK STABILITY BEFORE USE. PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS. CHILD AND TODDLER USE UNDER SUPERVISION ONLY. HANDLES ARE FOR SUPPORT ONLY AND ARE NOT LOAD BEARING TO THE SAME WEIGHT AS THE STEP.

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