GORDON
ELLIS
& CO

TIMBERRY KURVE

COMMODE CHAIR

INSTRUCTIONS FOR USE

en

6

cs

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8

de

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10

fr

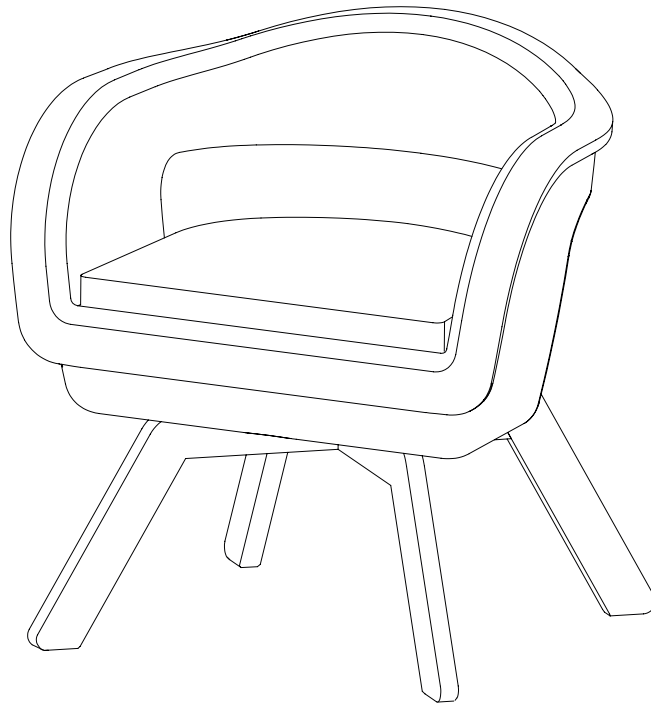
MODE D'EMPLOI

13

nl

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15

REF 55500, 56551, 56552, 56553, 56554, 56555, 56556, 56557, 56558, 56559, 56560

EU REP

Advena Ltd.

Tower Business Centre, 2nd Floor, Tower Street, Swatar. MALTA. BKR 4013

Gordon Ellis & Co.

Trent Lane, Castle Donington, Derbys. UK. DE74 2AT

56551-0926-5

UK
CA

CE

MD

ISS DATE:
SEPTEMBER 2026

GORDONELLISHEALTHCARE.CO.UK

en Accessible format available on request.
bg Достъпен формат е наличен при поискване.
cs Přístupný formát je k dispozici na vyžádání.
da Tilgængeligt format fås efter anmodning.
de Barrierefreies Format auf Anfrage erhältlich.
el Διαθέσιμη προσβάσιμη μορφή κατόπιν αιτήματος.
es Formato accesible disponible a petición.
et Juurdepääsetav vorming on saadaval nõudmisel.
fi Saavutettava versio saatavilla pyynnöstä.
fr Format accessible disponible sur demande.
hr Dostupan format na zahtjev.

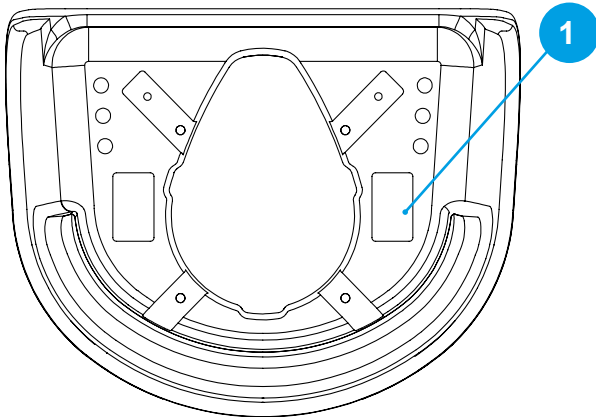
hu Hozzáférhető formátum kérésre elérhető.
it Formato accessibile disponibile su richiesta.
lt Prieinamas formatas pagal pareikalavimą.
lv Pieejams pielāgots formāts pēc pieprasījuma.
nl Toegankelijk formaat op verzoek beschikbaar.
no Tilgjengelig format tilgjengelig på forespørsel.
pl Dostępny format na życzenie.
pt Formato acessível disponível mediante pedido.
ro Format accesibil disponibil la cerere.
sk Dostupný prístupný formát na požiadanie.
sl Na zahtevo je na voljo dostopna oblika.
sv Tillgängligt format finns på begäran.

1

a.


b.

c.

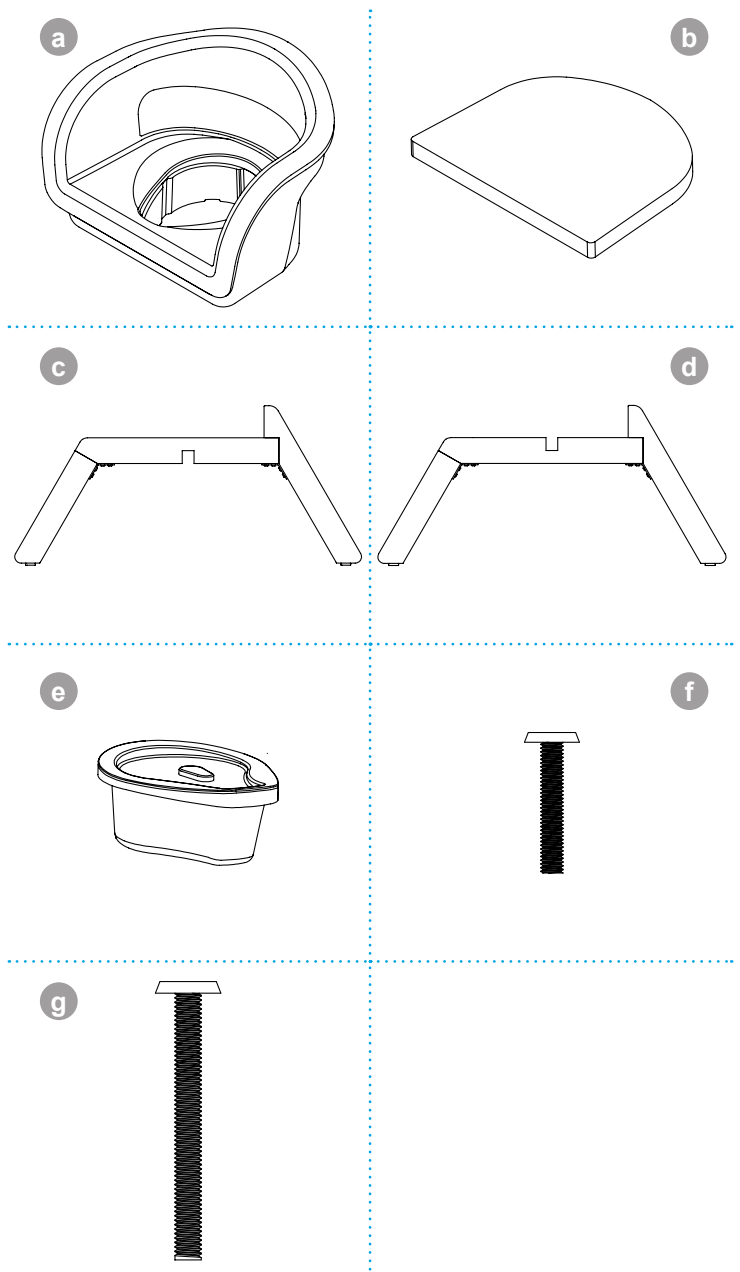
d.


e.

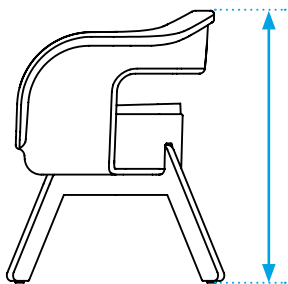1


73 cm (28.7")

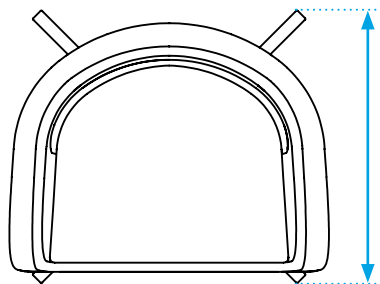2


59 cm (23.2")

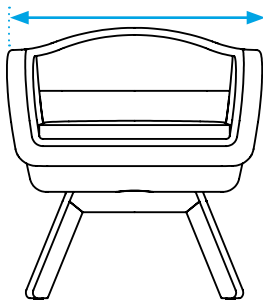3


69 cm (27.2")

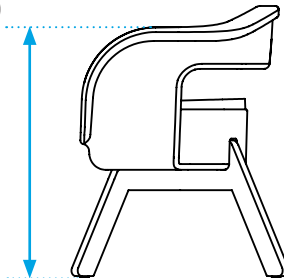4


67 cm (26.4")

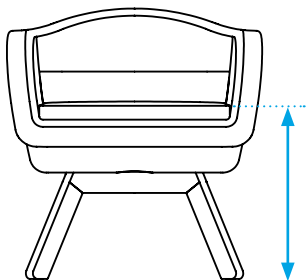5


47 cm (18.5")

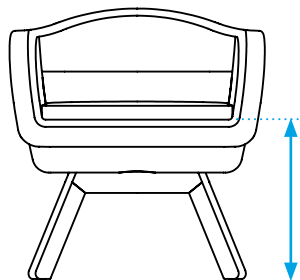6


43 cm (16.9")

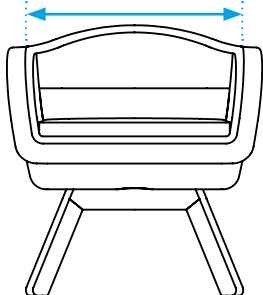7


54 cm (21.3")

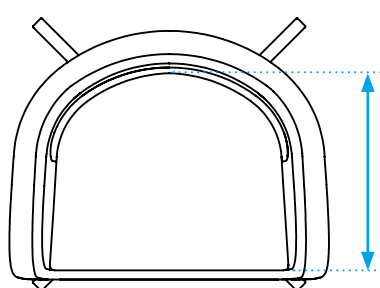8


45 cm (17.7")

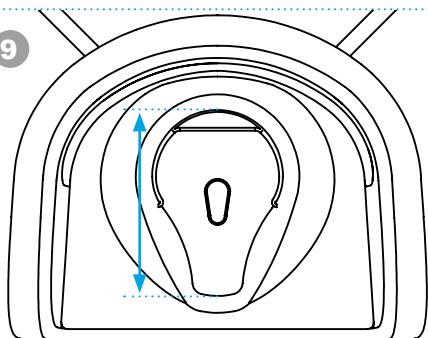9


31 cm (12.2")

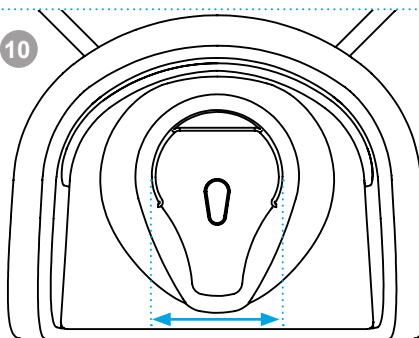10

22 cm (8.7")

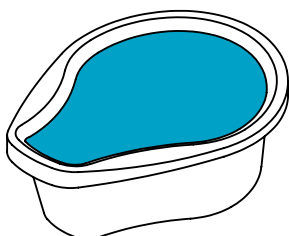11






Max.
3.5 L

12

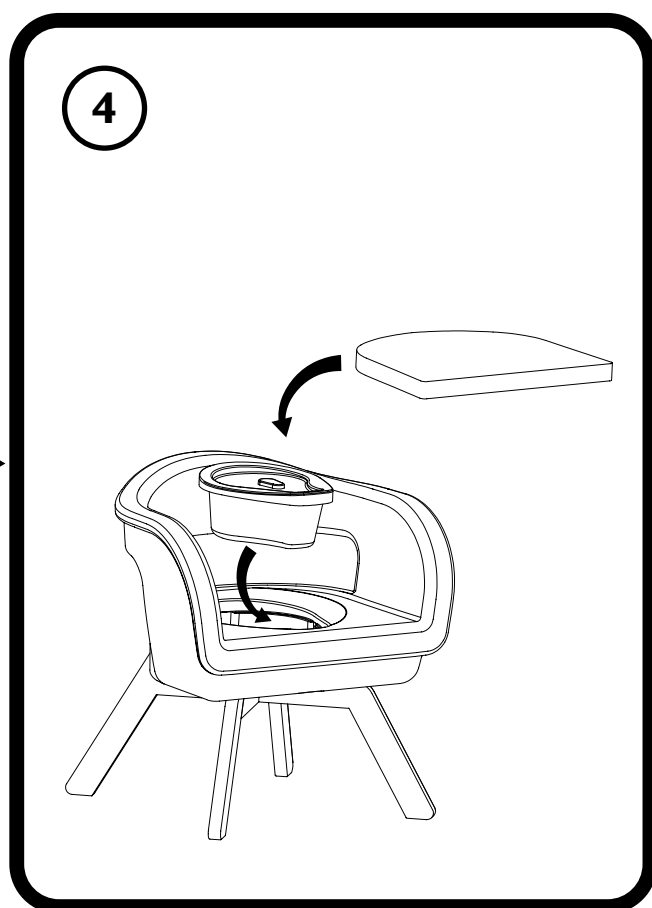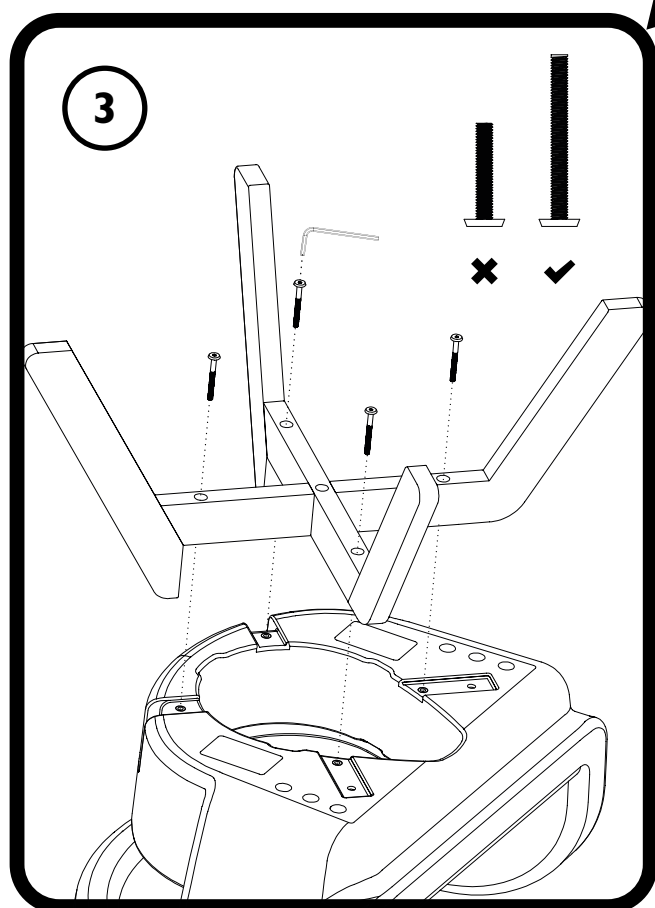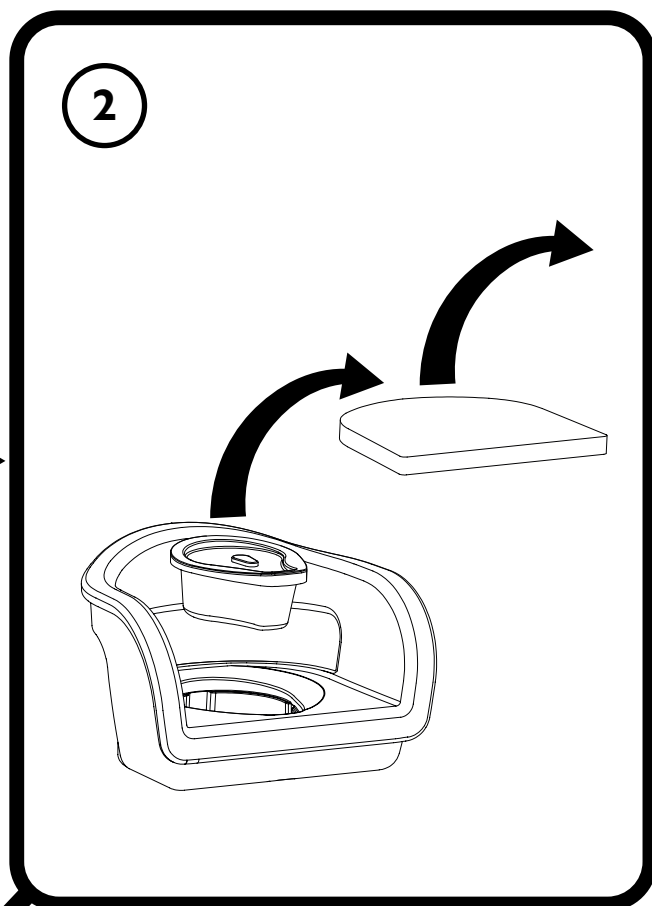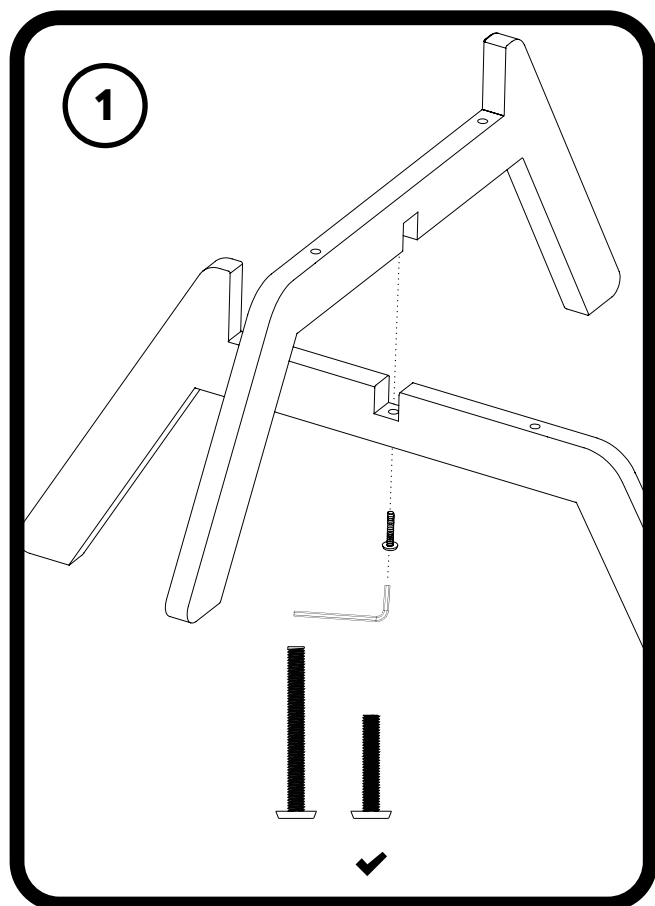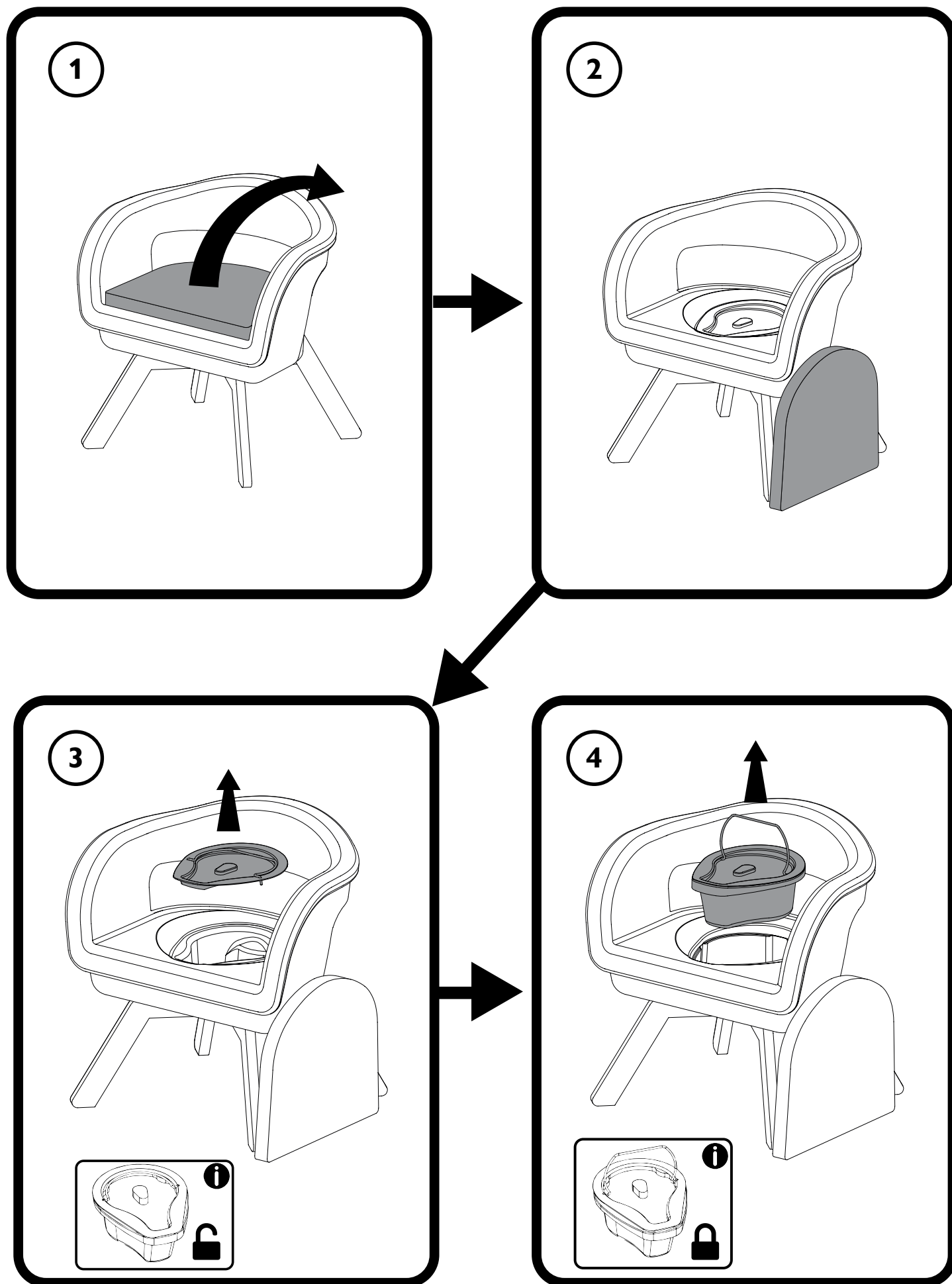Max.
190 Kg
(420 lbs)

Prior to reading these instructions for use, please check you have the latest version of this document, you can check the latest PDF version online at: <https://www.gordonellishealthcare.co.uk/timberly-kurve-commode>

Here you will also find other product documentation.

Introduction

Welcome to your new Gordon Ellis product. We hope that it proves a useful aid in your daily life. Our UK design team have developed your Timberly Kurve Commode Chair specifically with the intention of compensating for an injury or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

These instructions for use contain important information which should be followed to ensure your safety while using this product. Please ensure these instructions are saved for future reference.

Any serious accident or adverse event occurring through use of the device should be reported in a timely manner to your local distributor and the competent authority in your country. The local distributor will forward information to the manufacturer.

Device description

The Timberly Kurve Commode is a free-standing, furniture-style commode chair designed to provide discreet, comfortable toileting support within domestic environments to alleviate difficulty in toileting by providing a stable, accessible platform, thereby reducing the risk of falls. It combines the appearance of traditional bedroom furniture with the functionality of a portable commode, making it suitable for use in bedrooms, living spaces, or areas where access to a conventional toilet is limited.

The device features a robust wooden frame with an integrated Polyethylene, backrest, and armrests to provide stability and support for users when sitting down and standing up. The upholstered seat is designed for comfort during extended periods of sitting, while also allowing the product to blend with home furnishings.

The commode includes a concealed removable pan located beneath the seat, enabling use independent of a fixed toilet. The pan can be easily accessed for emptying and cleaning. When not in use as a commode, the unit functions as a standard chair, with the toileting function discreetly hidden to maintain dignity and minimise the clinical appearance of the product.

Intended use

Intended Purpose

To provide a free-standing, supportive seating solution for toileting.

Indications for Use

For individuals with reduced mobility, strength, balance, or endurance who have difficulty accessing or using a conventional fixed toilet.

Intended users

Lay persons or caregivers.

Intended Environment

Domestic or residential care environments.

Contraindication

The use of the device should not be used in the following situations:

- Users unable to safely bear weight or stabilise themselves, even with arm support.
- Users who cannot lower themselves into a seated position without significant risk of falling, even with assistance.
- Users requiring a height-adjustable or specialist seating system to achieve a safe or

clinically appropriate posture.

- Environments where the floor surface is unstable, sloped, soft, or uneven, preventing safe contact with all four legs.
- Users experiencing dizziness, fainting, or sudden loss of coordination, which may compromise safe toileting.
- Users requiring a load-bearing support aid for pulling themselves up from the front – this device is not designed for that use.
- Use in baths, showers, or other wet areas where slipping or instability may occur (unless specifically risk-assessed as safe).

Risk of injury

Please do not use the device and seek help from your local dealer, healthcare professional or provider if any of the following apply:

- If the seat moulding, legs, or upholstered seat appear damaged, loose, worn, or unstable.
- There are signs of fluid ingress, contamination, or deterioration that may compromise hygiene or material integrity.
- The device has been subjected to impact, misuse, or loading beyond its intended use.
- The device rocks when placed on a level floor.

Compliance

The manufacturer Gordon Ellis & Co. have a full quality management system accredited by the British Standards Institution to ISO 9001:2015.

EU Declaration of Conformity

This device meets the requirements of the Medical Device Regulation 2017/745, as a Class 1 medical device. The Declaration of Conformity document can be viewed online here: <https://www.gordonellishealthcare.co.uk/timberly-kurve-commode>

Any serious incident arising from use of the device should be reported in a timely manner to the European Authorised Representative (for EU customers) or directly to the manufacturer.

Device testing standards

This device has been tested and meets the requirements of:

- BS EN 21856:2022 - Assistive products - General requirements and test methods
- EN ISO 14971:2019+A2:2021 - Application of risk management to medical devices (risk-assessed)

Expected service life

The expected service life of this product is 5 years, assuming daily use and correct installation, adjustment, cleaning, and maintenance in accordance with this document.

Storage:

Store indoors in a clean, dry environment ideally between 10°C and 30°C.

Avoid:

- High humidity (above 60%)
- Freezing temperatures
- Excessive heat
- Prolonged direct sunlight
- Damp storage areas

If stored for more than 6 months, the device should be re-inspected before re-issue.

Limitation of liability, and customisation

The manufacturer accepts no liability for damage or injury arising from:

- Customisation or modification: Any alteration to the device - including structural changes, use of non-original spare parts, or unauthorised repairs - renders the device non-compliant and may compromise user safety.
- Improper use: Including use beyond the intended purpose, incorrect height adjustment, use on unstable surfaces, or use as a load-bearing mobility aid.
- Reasonable deterioration of components consistent with expected daily use.

In the event of contravening point a. or b. the device will no longer retain its CE marking.

Safety information and warnings

Do not use the device until these instructions have been fully read and understood.

If anything is unclear, contact your dealer or healthcare professional before use.

Risk of serious injury or falls

- Do not use the device if any part appears loose, damaged, cracked, or unstable.
- Check that the device sits flat and stable on a level surface before each use.
- Do not use on slopes, uneven, or unstable ground.
- Do not exceed the maximum user weight indicated on the label.
- Do not drop heavily onto the seat. Sit in a controlled manner.
- Do not use the device as a step, ladder, stool, or load-bearing support.
- Ensure the Ashby pan is correctly positioned and fully supported before use.
- Do not use the device if it rocks or moves during use.
- Do not lean heavily sideways on the arms or apply forward pulling forces.

Risk of instability

- Ensure the Ashby pan is correctly positioned and fully supported before use.
- Check that all feet are in full contact with the floor before sitting down.
- Do not attempt to move or reposition the device while someone is sitting on it.
- Ensure the device is positioned where there is sufficient space for safe transfer on and off the chair.
- Do not place the device on loose rugs or slippery surfaces unless suitably secured.

Risk of injury

- Use only original spare parts and accessories supplied or approved by the manufacturer.
- Do not attempt repairs yourself.
- Take care when handling or removing the commode pan to avoid spillage or contamination.
- Remove from use if instability or unusual movement is detected

Instructions, device, and label symbols

Instruction symbols: (Image reference: 1a.)

Important information, useful advice.

	Warning, limitation, or caution.
	Maximum user weight

Label symbols: (Image reference: 1c.)

1	Unique Device Identification (UDI)
2	Manufacturer
3	Country of manufacture
4	Medical device
5	European Conformity mark
6	UK Conformity Assessed mark
7	Lot no. / production date (YY-MM-DD)
8	European Authorised Representative
9	Triman Symbol (France only)
10	European Article Number (EAN)
11	Product linear code-128
12	Product description
13	Product reference number
14	Instructions for use symbol

 Retain the device label for future reference.

Device components

(Image reference: 1d.)

Ref.	Component	Product
a	Seat moulding	All
b	Upholstered seat	56551-56560
c	Leg 1	All
d	Leg 2	All
e	Ashby Pan and Lid	All
f	M6x35mm Screw	All
g	M6x60mm Screw	All

Assembly and installation (pg. 4 & 5)

Before assembly, ensure all components are present and that no part is damaged or missing. If any defect is identified, do not use the device and contact your dealer or healthcare provider.

2a. Assembly Instructions

- Slot the legs together and secure using the M6x35mm screw.
- Remove the upholstered seat and Ashby pan.
- Turn the seat moulding over and secure the legs at the four fixing points using the M6x60mm screws.
- Return the Ashby pan and upholstered seat.

2b. Commode Instructions

- Remove the upholstered seat.
- Store the upholstered seat in a safe location while using as a commode.
- Remove the lid from the Ashby pan - ensure the handle remains down to allow removal of the lid only. Use the commode.
- Return the lid. Lift the pan from the commode ensuring the handle is vertical to lock the lid. Following waste disposal, return the pan and upholstered seat.

Safety notes (Assembly & Installation)

- Only use the device on a firm, level surface.
- Do not use if any one of the leg, seats, or Ashby pan appears loose or damaged.

Safety notes (Assembly & Installation)

- Do not attempt to modify or alter any structural component.

Installer / user responsibility

It is the responsibility of the user or installer to:

- Carry out an initial local risk assessment
- Ensure the device is stable before use
- Re-check all adjustments after cleaning, relocation, or re-issue
- Remove the device from service if instability or wear is detected

Technical Information

Device Symbols: (Image reference: 1b.)

1	Product information label location
---	------------------------------------

Symbol	Description
	EU Authorised Representative
	Manufacturer
	LOT Number
	European Conformity symbol
	UK Conformity Assessed mark
	Device Reference number
	Medical device symbol
	Instructions for use
	Unique device identifier
	Maximum user weight

Device Materials: (Image reference: 1d.)

a	MDPE
b	Chipboard, Foam, Woven Fabric
c, d	Beech, Stainless Steel
e	PP
f, g	Mild Steel (BZP)

This device and component parts are manufactured from corrosion-resistant materials, and do not contain natural rubber or latex.

Device dimensions: (Image reference: 1e.)

#	Description
1	Total height
2	Total depth
3	Total width
4	Armrest height from floor
5	Seat height (with upholstered seat)
6	Seat height (without upholstered seat)
7	Width between arms
8	Seat depth
9	Aperture depth
10	Aperture width
11	Maximum pan volume
12	Maximum user weight

Cleaning, maintenance, re-issuing, and disposal

Risk of contamination

- When cleaning or handling a used commode, especially the commode pan, always follow appropriate hygiene precautions.
- Wear disposable gloves, ensure adequate ventilation, and follow the instructions of any cleaning product used.

To maintain hygiene and preserve the service life of the device, follow the cleaning instructions below.

DO NOT use abrasive cleaners, wire brushes, scouring pads, or strong chemical disinfectants (e.g., bleach-based or solvent-based cleaners such as acetone or toluene), as these may damage the surface finish, weaken components, or cause corrosion.

ONLY use mild soap or a neutral, non-abrasive disinfectant cleaner suitable for metal and plastic surfaces. Always follow the cleaner manufacturer's instructions

AVOID prolonged soaking, submersion, or exposure to high-pressure jets of water.

Failure to follow these instructions may lead to corrosion, surface degradation, or reduced safety of the product.

Recommended Cleaning Process

Routine Daily Cleaning

(For general hygiene, household use):

- Remove any visible contamination from the seat, armrests, and frame using warm soapy water.
- Wipe all accessible surfaces (including the underside of the seat) with a damp cloth or sponge.
- Rinse with clean water to remove any detergent residue.
- Dry thoroughly using a clean, lint-free cloth - paying particular attention to joints.
- Ensure the device is completely dry before use or storage.

Cleaning the Commode Pan

- Remove the pan carefully from the seat moulding.
- Empty the contents into a toilet following local hygiene guidelines.
- Wash the pan using warm water and detergent, ensuring all residues are removed.
- Disinfect using a neutral, non-abrasive disinfectant.
- Rinse thoroughly and dry before replacing onto the support bracket.

Enhanced Disinfection

(After contamination or clinical use)

Use a non-abrasive disinfectant cleaner recommended for use on medical or assistive products. Follow the manufacturer's contact time and wipe away residues with clean water. Dry thoroughly before reuse.

Maintenance and Inspection Checklist

Before each use and during periodic inspection, ensure the following:

Inspection Point	Check For	Action if fault found
Seat Moulding	Cracks, bends, deformation, loose fixing	Remove from use immediately and replace.

Inspection Point	Check For	Action if fault found
Wooden Legs	Cracks, splits, loose joints, structural damage	Remove from use; replace or repair using approved parts.
Upholstered Seat	Tears, cracks, sagging, loose covering, loss of support	Remove from use immediately and replace or repair using approved parts
Pan & Bracket (Commode)	Loose fittings, instability	Refit or replace bracket/pan.
Overall stability	Rocking, movement on level floor	Re-adjust legs; if instability persists, remove from use.

Routine inspection should be carried out at least every 6 months, or before re-issue. Record the inspection date and findings if required by local procedures.

Re-issuing:

This device is suitable for re-issue, provided the following conditions are met:

-  A full inspection has been carried out using the checklist above
-  High-level cleaning and disinfection have been completed
-  Any worn or damaged components have been replaced using original spare parts
-  The seat and commode pan (if applicable) are undamaged and securely fitting

If any component fails inspection and suitable replacement parts are not available, **do not re-issue the device.**

Disposal:

Where possible, disassemble the device into its material groups to support recycling.

This device can be disposed of at many recycling facilities, please check the following for appropriate options in your locality:

- Check your local authority recycling guidelines for plastic and wooden goods or mixed materials.
- Separate materials where possible to aid recycling.

-  If the device has been used in a healthcare or clinical environment and is considered contaminated, it must be disposed of according to your region's clinical waste regulations.

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