EN PURPOSE DESIGNED FOR IMPROVED BATHROOM SAFETY WITH A TEXTURED FINISH FOR EXTRA GRIP. ENSURE WALL IS LEVEL. DRILL THROUGH AT LEAST TWO OF THE BLANKED SCREW HOLES IN THE FIXING BRACKETS, BOTH ENDS. ENSURE FIRM FIXING BEFORE EACH USE. CONSULT A TRADESMAN IF UNSURE ABOUT WALL FIXING - SIZE TO WALL PLUGS RECOMMENDED FOR NORMAL BRICKWORK. IF MOVED, REFOLLOW ALL STEPS TO ENSURE SAFE USE. NOT RECOMMENDED FOR OUTDOOR USE, OR AREAS EXPOSED TO DIRECT SUNLIGHT.

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12, 18, 24 INCH STRAIGHT GRAB BAR

13 INCH ANGLED GRAB BAR

Welcome to your new Gordon Ellis product.
 We hope that it proves a useful aid in your daily life.


SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS
 AVAILABLE IN THE FOLLOWING LANGUAGES:
 EN FR DE IT ES NL PT DK SV NO FI EL

MAXIMUM USER WEIGHT	POIDS MAXIMUM DE L'UTILISATEUR	PESO MÁXIMO DEL USUARIO
MAX 115 kg 254lbs	MAXIMALES BENUTZERGEGWICHT	MAX. BRUGGERVÆGT
MAXIMUMGEWICHT VAN GEBRUIKER	PESO MÁXIMO DE USO	MAKS. BRUGGERVÆGT
HÖGSTA ANVÄNDARVIKT	MAKSIMUM BRUKERVEKT	SUURIN KÄYTTÖPAINO
ΑΝΩΤΑΤΟ ΟΠΙΟ ΒΑΡΟΥΣ ΤΟΥ ΧΡΗΣΤΗ		


EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER - CONTACT DETAILS BELOW
FR TOUT ACCIDENT GRAVE SURVENANT PENDANT L'UTILISATION DOIT ÊTRE SIGNALÉ AU EC REP - COORDONNÉES CI-DESSOUS
DE JEDER SCHWERE UNFALL WÄHREND DES GEBRAUCHS MUSS EC REP GEMELDET WERDEN - KONTAKTDATEN UNTEN
IT QUALSIASI INCIDENTE GRAVE CHE SI VERIFICA DURANTE L'USO DEVE ESSERE SEGNALATO AL PRODUTTORE - DETAGLI DI CONTATTO DI SEGUITO
ES CUALQUIER ACCIDENTE GRAVE QUE OCURRA DURANTE EL USO DEBE INFORMARSE A EC REP - DETALLES DE CONTACTO A CONTINUACIÓN
NL ELK ERNSTIG ONGEVAL TIJDENS HET GEBRUIK MOET WORDEN GEMELD AAN EC REP - CONTACTGEGEVENS HIERONDER
PT QUALQUER INCIDENTE SÉRIO QUE OCORRA DURANTE O USO DEVE SER RELATADO AO FABRICANTE - DETALHES DE CONTACTO ABAIXO
DK ENHVER ALVORLIG HENDELSE, DER OPSTÅR UNDER BRUG, SKAL RAPPORTERES TIL PRODUCENTEN - KONTAKTOPPLYSNINGER NEDENFOR
SV ALLA ALLVARLIGA HÄNDELSER SOM INTRÄFFAR UNDER ANVÄNDNING MÅSTE RAPPORTERAS TILL TILLVERKAREN - KONTAKTUPPGIFTER NEDAN
NO ENHVER ALVORLIG HENDELSE SOM OPPTÅR UNDER BRUK MÅ RAPPORTERES TIL PRODUSENTEN - KONTAKTINFORMASJON NEDENFOR
FI KAIKISTA KÄYTON AIKANA TAPAHTUNEISTA VAKAVISTA TAPAHTUMISTA ON ILMOITETTAVA VALMISTAJALLE - YHTYESTIEDOT ALLA
EL ΚΑΘΕ ΣΟΒΑΡΟ ΣΥΜΒΑΝ ΠΟΥ ΕΜΦΑΝΙΖΕΤΑΙ ΚΑΤΑ ΤΗ ΧΡΗΣΗ ΠΡΕΠΕΙ ΝΑ ΑΝΑΦΕΡΕΤΑΙ ΣΤΟΝ ΚΑΤΑΧΕΥΣΤΗ - ΠΑΡΑΚΑΤΩ ΣΤΟΙΧΕΙΑ ΕΠΙΚΟΙΝΩΝΙΑΣ

Your ASHBY GRAB BARS were specifically developed for our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

Ai ASHBY STRAIGHT GRAB BAR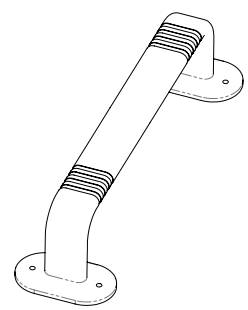

Bi ASHBY ANGLED GRAB BAR

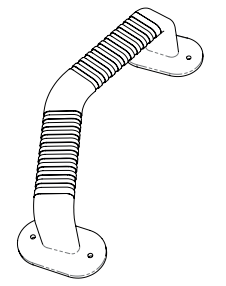CLEANING INSTRUCTIONS	INSTRUCTIONS DE NETTOYAGE	REINIGUNGSANWEISUNGEN
ISTRUZIONI PER LA PULIZIA	INSTRUCCIONES DE LIMPIEZA	SCHOONMAAK INSTRUCTIES
INSTRUÇÕES DE LIMPEZA	RENGØRINGSINSTRUKTIONER	RENGØRINGSINSTRUKTIONER
RENGÖRINGSINSTRUKSJONER	PUHDISTUSOHJEET	ΟΔΗΓΙΕΣ ΚΑΘΑΡΙΣΜΟΥ

PH 5 - 9

EN WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.
FR ESSUYEZ AVEC UN CHIFFON NON ABRASIF ET DE L'EAU SAVONNEUSE.
DE MIT EINEM NICHT SCHEUERNDEN TUCH UND SEIFENWASSER ABWISCHEN.
IT PULIRE CON UN PANNON NON ABRASIVO E ACQUA SAPONOSA.
ES LIMPIELO CON UN PAÑO NO ABRASIVO Y AGUA CON JABÓN.
NL AFWENEM MET EEN NIET-SCHURENDE DOEK EN ZEEPACHTIG WATER.
PT RETIRAR COM PAÑO NÃO ABRASIVO E ÁGUA COM SABÃO.
DK AFTØRRER MED EN IKKE-SLIPENDE KLUD OG SÆBEVAND.
SV TORKA AV MED EN IKKE-SLIPANDE TRASA OCH TVÄLVATTEN.
NO TØRK AV MED EN IKKE-SLIPENDE KLUT OG SÅPEVANN.
FI PYYHI HANKAAMATTOMALLA LIINALLA JA SAIPUVAIVEDELLÄ.
EL ΣΚΟΥΠΙΣΤΕ ΜΕ ΕΝΑ ΜΗ ΛΕΙΑΝΤΙΚΟ ΠΑΝΙ ΚΑΙ ΣΑΙΠΩΝΟΝΕΡΟ.

DESIGNED & MADE IN THE UK | BHTA | 7 ABS | RECYCLED | EC REP






GORDON ELLIS & CO.
 TRENT LANE
 CASTLE DONINGTON
 DERBY, ENGLAND DE74 2AT

AUTHORISED REPRESENTATIVE
 ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

MD | IFU-REF | 6218/B/1020-1 | PART REF | 6218/B + 6219/B + 6220/B + 6257/B + 6268/B + 6278/B + 6269/B + 6279/B + 6270/B + 6280/B

ASHBY STRAIGHT GRAB BAR

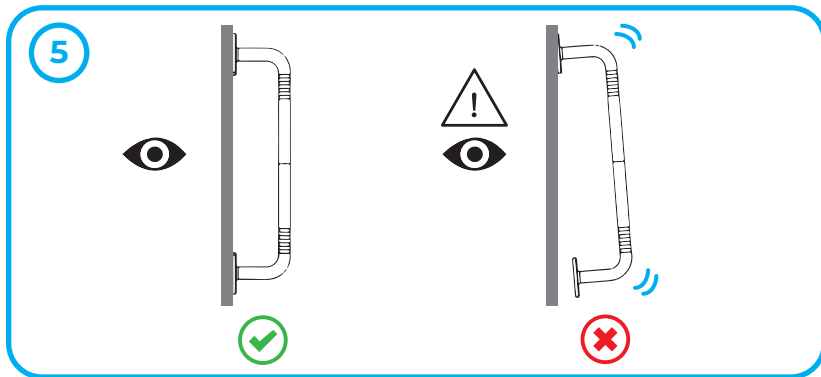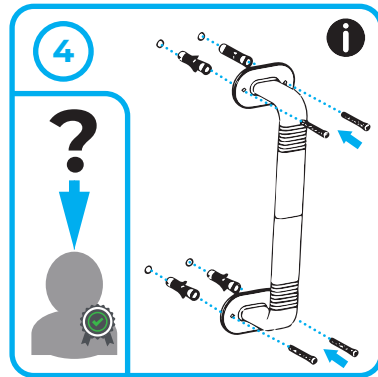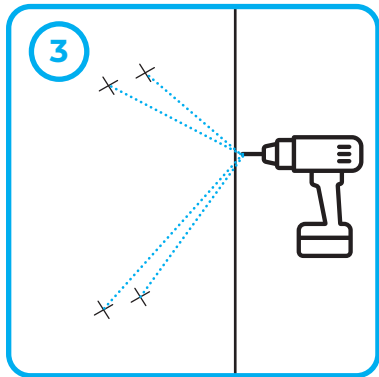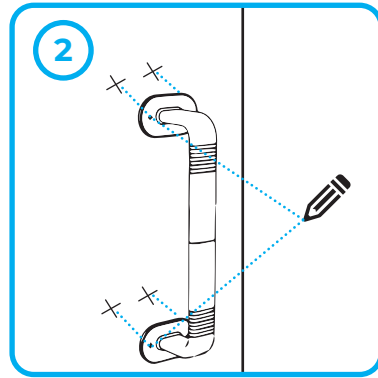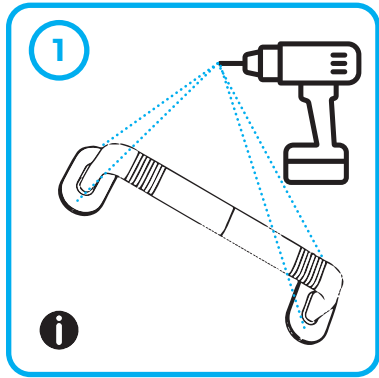ASHBY ANGLED GRAB BAR