



BIG FOOT® HALF STEP LINKING KIT

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS AVAILABLE IN THE FOLLOWING LANGUAGES: EN

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER

GORDON
ELLIS
& CO

SUITABLE FOR: BIG FOOT HALF STEP
66111




DESIGNED AND MADE IN THE UK



IFU-REF 61097/1020-1

PART REF 61097

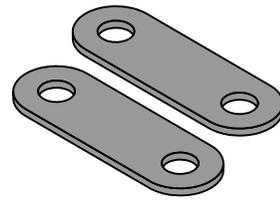

KIT CONTENTS:
TWO METAL LINKING BARS




PLEASE READ THESE INSTRUCTIONS ALONG WITH THE INSTRUCTIONS INCLUDED WITH THE BIG FOOT HALF STEP.

ENSURE THE STEP IS STABLE BEFORE USE.


PERIODICALLY CHECK AND RE-TIGHTEN ALL NUTS AND OTHER FIXINGS AS REQUIRED.

FIXING INSTRUCTIONS:

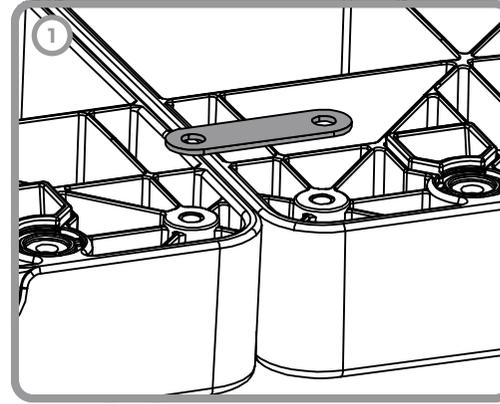

PLACE THE TWO STEPS BEING LINKED NEXT TO EACH OTHER, UPSIDE DOWN. REMOVE THE FOUR FEET WHERE THE STEPS WILL BE LINKED.

(1) LOCATE A LINKING BAR OVER THE MATCHING HOLES ON EACH STEP. REPEAT FOR THE OTHER TWO HOLES ON THE OTHER SIDE OF THE STEP.

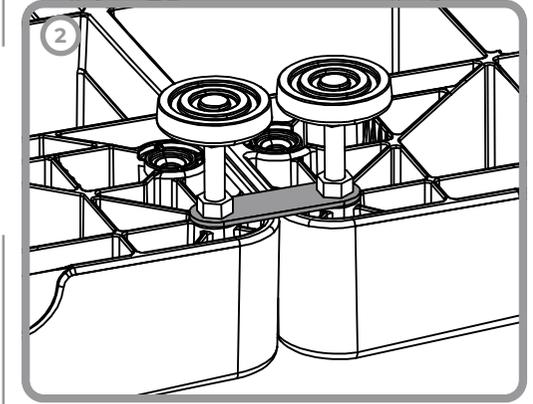

(2) SCREW FEET THROUGH THE LINKING BAR AND TIGHTEN UP SECURELY SO THE THE LOCKING NUT IS FIRMLY AGAINST THE LINKING BAR.

CONT. OVERLEAF



BIG FOOT® HALF STEP LINKING KIT

SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS AVAILABLE IN THE FOLLOWING LANGUAGES: EN

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE MANUFACTURER

GORDON
ELLIS
& CO

SUITABLE FOR: BIG FOOT HALF STEP
66111




DESIGNED AND MADE IN THE UK



IFU-REF 61097/1020-1

PART REF 61097

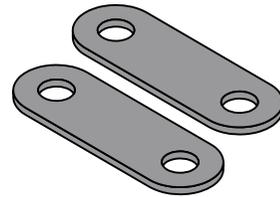

KIT CONTENTS:
TWO METAL LINKING BARS




PLEASE READ THESE INSTRUCTIONS ALONG WITH THE INSTRUCTIONS INCLUDED WITH THE BIG FOOT HALF STEP.

ENSURE THE STEP IS STABLE BEFORE USE.


PERIODICALLY CHECK AND RE-TIGHTEN ALL NUTS AND OTHER FIXINGS AS REQUIRED.

FIXING INSTRUCTIONS:

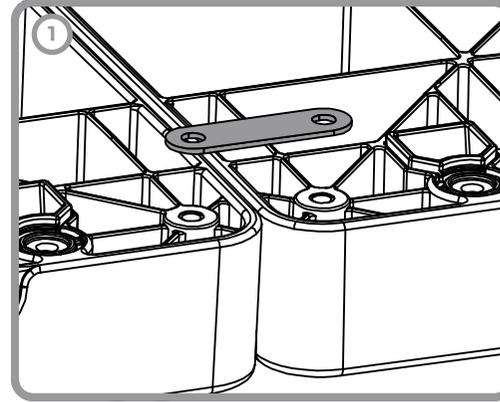


PLACE THE TWO STEPS BEING LINKED NEXT TO EACH OTHER, UPSIDE DOWN. REMOVE THE FOUR FEET WHERE THE STEPS WILL BE LINKED.

(1) LOCATE A LINKING BAR OVER THE MATCHING HOLES ON EACH STEP. REPEAT FOR THE OTHER TWO HOLES ON THE OTHER SIDE OF THE STEP.

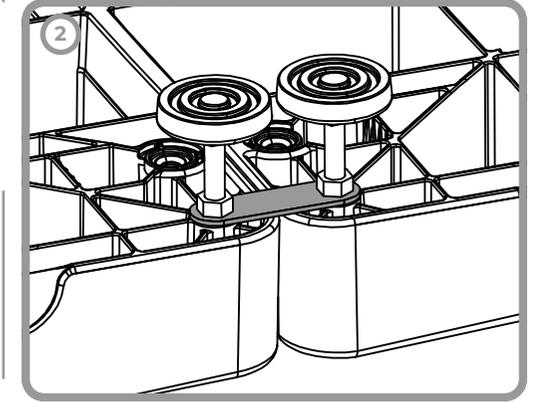


(2) SCREW FEET THROUGH THE LINKING BAR AND TIGHTEN UP SECURELY SO THE THE LOCKING NUT IS FIRMLY AGAINST THE LINKING BAR.

CONT. OVERLEAF

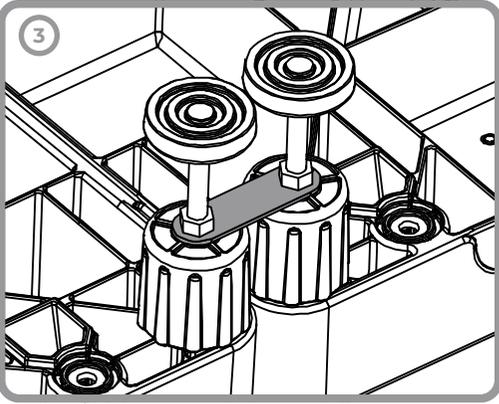

(3) IF USING WITH THE EXTENDING FOOT FEATURE ENSURE THE LINK BAR IS BETWEEN THE BOTTOM OF THE EXTENDING FOOT AND THE LOCKING NUT.

LINKING KIT + FIXING KIT:

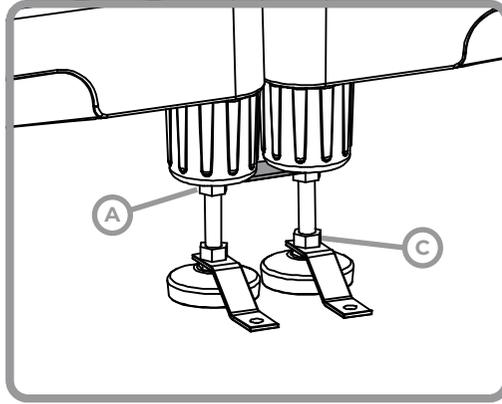

PLEASE READ WITH 'SECURITY FIXING KIT' INSTRUCTIONS SUPPLIED WITH BIG FOOT HALF STEP USER GUIDE.

THE LINKING KIT GOES ABOVE THE LOCKING NUT (A)

THE FIXING KIT GOES BELOW THE LOCKING NUT (C)

SAFETY CONSIDERATIONS:



AT LEAST 1.5 cm OF THREAD MUST REMAIN INSIDE THE ADJUSTING LEG (AND EXTENDING LEG, IF USING) FOR SAFE USE.

ENSURE BOTH STEPS ARE SECURELY JOINED AND STABLE BEFORE USE.

IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS.

PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS.

NOT FOR USE WITH ANY OTHER STEPS. NOT FOR ANY OTHER USE THAN DESCRIBED.

CLEANING INSTRUCTIONS




PH 5 - 9

WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.




GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK
f gordonellis.healthcare @EllisHealthcare

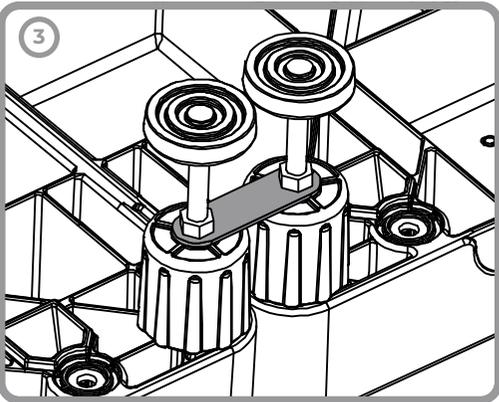

(3) IF USING WITH THE EXTENDING FOOT FEATURE ENSURE THE LINK BAR IS BETWEEN THE BOTTOM OF THE EXTENDING FOOT AND THE LOCKING NUT.

LINKING KIT + FIXING KIT:

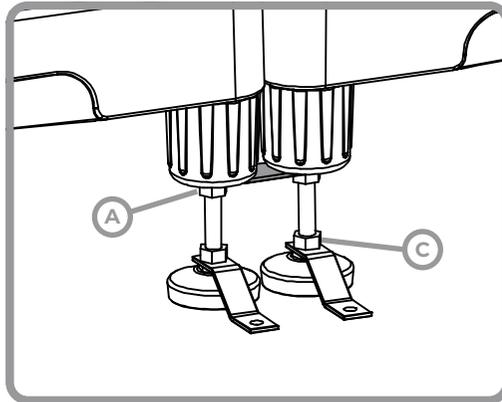

PLEASE READ WITH 'SECURITY FIXING KIT' INSTRUCTIONS SUPPLIED WITH BIG FOOT HALF STEP USER GUIDE.

THE LINKING KIT GOES ABOVE THE LOCKING NUT (A)

THE FIXING KIT GOES BELOW THE LOCKING NUT (C)

SAFETY CONSIDERATIONS:



AT LEAST 1.5 cm OF THREAD MUST REMAIN INSIDE THE ADJUSTING LEG (AND EXTENDING LEG, IF USING) FOR SAFE USE.

ENSURE BOTH STEPS ARE SECURELY JOINED AND STABLE BEFORE USE.

IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS.

PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS.

NOT FOR USE WITH ANY OTHER STEPS. NOT FOR ANY OTHER USE THAN DESCRIBED.

CLEANING INSTRUCTIONS



PH 5 - 9

WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.



GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK
f gordonellis.healthcare @EllisHealthcare