



# NLP129 & NLP129S

## ADJUSTABLE BED RAISER

Welcome to your new Gordon Ellis product.  
We hope that it proves a useful aid in your daily life.



EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS OVER LEAF



SEE REVERSE FOR FITTING INSTRUCTIONS  
AVAILABLE IN THE FOLLOWING LANGUAGES:  
EN



MAXIMUM USER WEIGHT

Your NLP129 ADJUSTABLE BED RAISER was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.



### LP/NLP129

SUITABLE FOR: **DOUBLE BEDS**  
SUITABLE BED WIDTHS: 91.4 - 146 CM  
MAXIMUM BED LEG SIZE: 6.4 X 7.6 CM




### LP/NLP129S

SUITABLE FOR: **SINGLE BEDS**  
SUITABLE BED WIDTHS: 61 - 89 CM  
MAXIMUM BED LEG SIZE: 6.4 X 7.6 CM



### RAISING HEIGHT

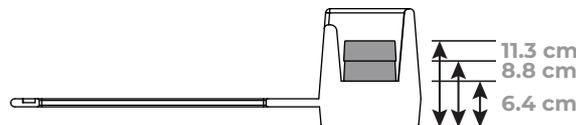

# LAN & HAM

INSPIRED BY YOUR NEEDS



**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY. ENGLAND DE74 2AT



**AUTHORISED REPRESENTATIVE**  
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA



MD

IFU-REF LP/NLP129/0121-1

PART REF LP/NLP129 + LP/NLP129S

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](http://gordonellis.healthcare)  
 @EllisHealthcare











# INSTRUCTIONS FOR USE

**i** EN WELL KNOWN AND TRUSTED THROUGHOUT THE INDUSTRY, THE LANGHAM ADJUSTABLE LINKED BED RAISER IS DESIGNED SPECIFICALLY FOR BEDS WITH LEGS, FITTING LEGS UP TO 6.4 X 7.6CM IN SIZE. EACH UNIT IS LINKED FOR ADDED STRENGTH WITH AN ELONGATED FOOTPRINT TO GIVE GREATER STABILITY AND REASSURANCE TO THE USER.

**FITTING INSTRUCTIONS:**

1. POSITION UNITS ON THE FLOOR PARALLEL TO THE FRONT AND REAR OF BED.
2. LOOSEN THE WING NUTS.
3. ADJUST TO SIZE SO THAT BED LEGS WILL DROP INTO THE RAISER UNITS.
4. LIFT BED AND PLACE LEGS INTO SOCKETS. ENSURE THE RAILS RUN SIDE TO SIDE AS TO MINIMISE THE RISK OF A TRIP HAZARD.
5. PUSH RAISER UNITS TIGHT UP AGAINST THE CORNERS OF THE LEGS.
6. TIGHTEN THE WING NUTS.
7. RAISER BLOCKS CAN BE INSERTED AS NECESSARY TO FURTHER RAISE BY 24MM OR 48MM - MAXIMUM TWO PER RAISER UNIT. ENSURE EACH RAISER UNIT IS EQUAL IF EXTRA HEIGHT IS ADDED.
8. ENSURE THE BED AND RAISER UNITS ARE STABLE BEFORE EACH USE AND IF KNOCKED.

**!** DEVICE TO BE USED IN STATIC MODE ONLY. EXTRA CARE SHOULD BE TAKEN TO AVOID SUDDEN OR UNDUE MOVEMENT. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. DO NOT USE WITH ANY OTHER DEVICE UNLESS STATED. TAKE CARE TO AVOID ENTRAPMENT OF FINGERS IN ADJUSTMENT SLOTS. LEGS MUST MAKE CONTACT WITH THE BOTTOM OF THE WELL - DEVICE IN NOT LOAD BEARING AT THE TOP OF THE UNIT. FURNITURE MAY BE HEAVY - TAKE EXTRA CARE WHEN FITTING THE RAISER UNIT.

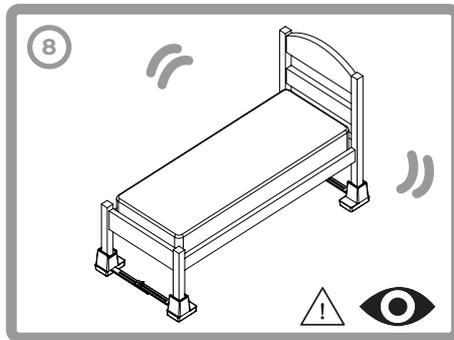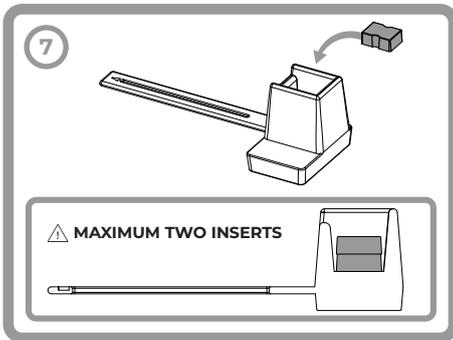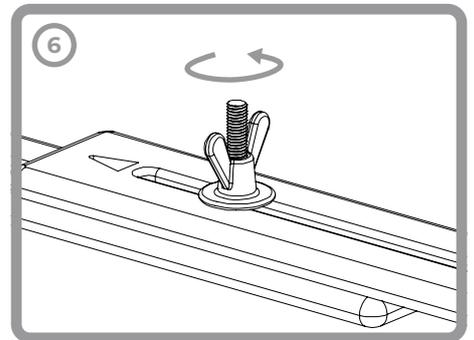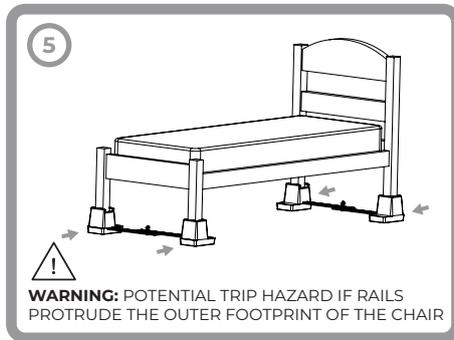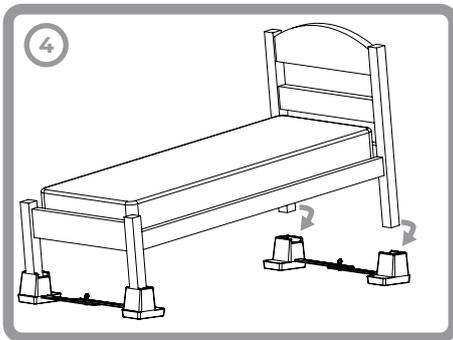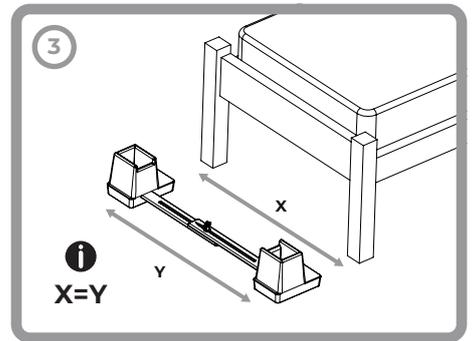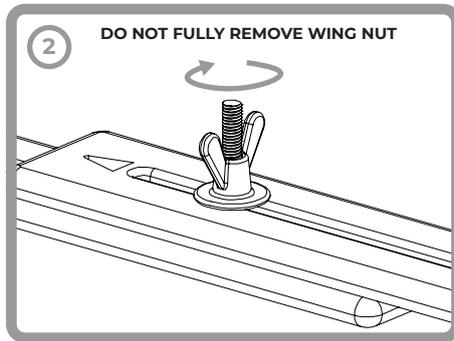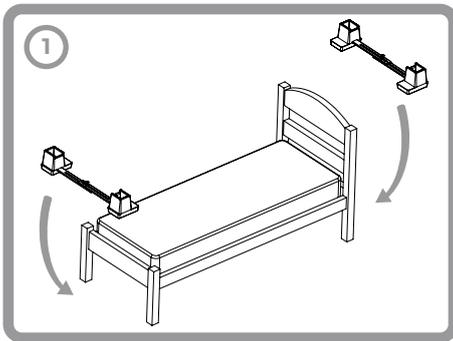

**i** **CLEANING INSTRUCTIONS**

**!** WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

DEVICE IS FULLY AUTOCLAVABLE.

**PH 5 - 9**