
GORDON  
ELLIS  
& CO

# BAMBERRY™ TRANSFER BOARDS


Welcome to your new Gordon Ellis product.  
We hope that it proves a useful aid in your daily life.

Your **BAMBERRY™ TRANSFER BOARDS** were specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.


## **i** 55935 - CURVED TRANSFER BOARD

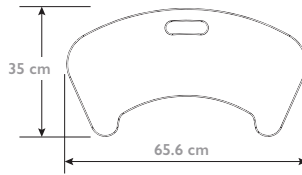## **i** 55932 - 30IN STRAIGHT TRANSFER BOARD
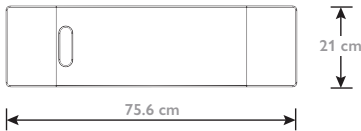
## **i** 55933 - 24IN STRAIGHT TRANSFER BOARD

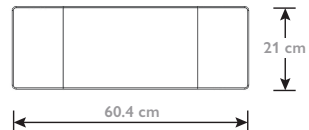DESIGNED  
IN THE UK

MADE IN  
CHINA

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY. ENGLAND DE74 2AT

EC REP

**AUTHORISED REPRESENTATIVE**  
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND  
FLOOR, TOWER STREET, SWATAR, BKR 4013  
MALTA

**i** MD


IFU-REF 55932/1023-3

REF 55932, 55933, 55935

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](http://gordonellis.healthcare)  
 [@EllisHealthcare](http://@EllisHealthcare)
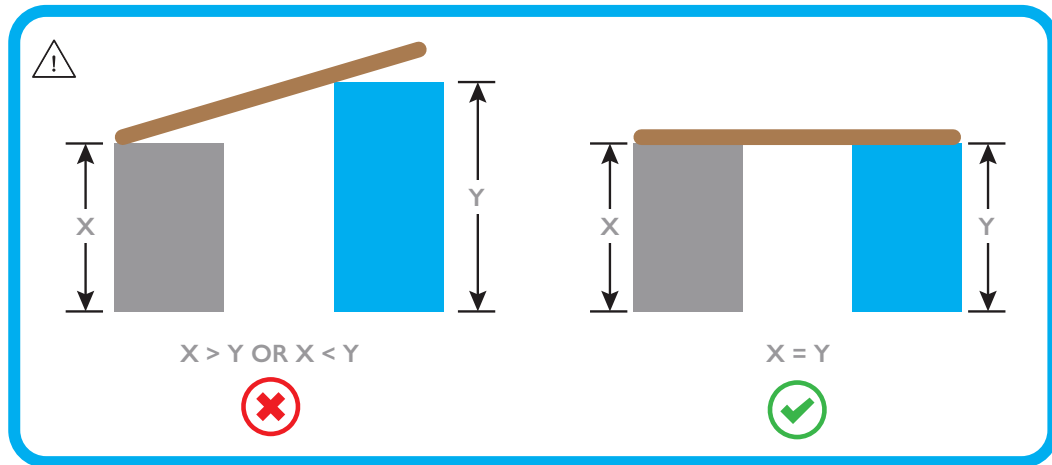


$X > Y$  OR  $X < Y$

$X = Y$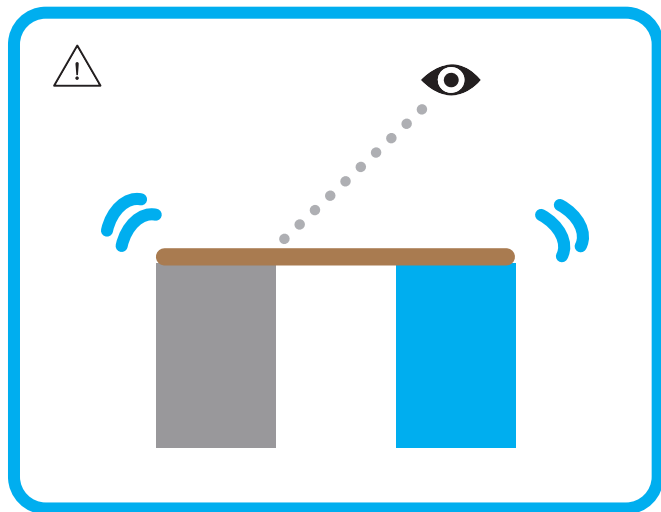

A

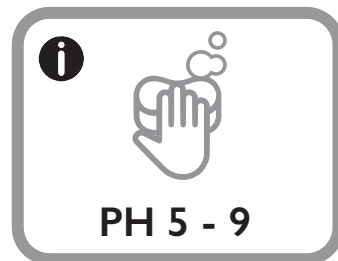B

EN

TRANSFER BOARDS ENABLE PEOPLE WITH LIMITED STRENGTH OR BALANCE TO TRANSFER THEMSELVES ON TO A SECURE SURFACE FROM A WHEELCHAIR TO A CAR, BED, CHAIR OR OTHER LOCATION INDEPENDENTLY AND SAFELY.

REDUCE FATIGUE FOR BOTH USER AND CARER. EVEN FOR PEOPLE WHO MAY ONLY HAVE TO PERFORM OCCASIONAL TRANSFERS, TRANSFER BOARDS OFFER AN ADDED MEASURE OF SAFETY WHEN A PERSON IS FATIGUED, ALONE IN THE HOUSE, OR IN UN-FAMILIAR SETTINGS.

1. PLACE THE TRANSFER BOARD IN TO POSITION BETWEEN TWO EQUAL HEIGHT SEATS.
2. CAREFULLY TRANSFER YOURSELF ON TO THE BOARD AND SLIDE ACROSS TO THE NEW SEATING AREA.

CHECK FOR STABILITY BEFORE EACH USE. IT IS THE RESPONSIBILITY OF THE OWNER/ USER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS BEFORE USE. DEVICE IS DESIGNED TO BE USED TO TRANSFER TO OR FROM POSITIONS OF A SIMILAR HEIGHT ONLY - NOT FOR ANY OTHER USE. IT IS ADVISED THE THE PRODUCT IS USED ONLY WHEN CLOTHED OR WITH A TOWEL IF NOT CLOTHED.

A = MAXIMUM USER WEIGHT.  
B = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY **SERIOUS ACCIDENT** OCCURRING THROUGH USE SHOULD BE REPORTED TO THE **EC REP** (EU CUSTOMERS) OR THE **MANUFACTURER** (UK & NON-EU CUSTOMERS) - **CONTACT DETAILS BELOW**

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