
TIMBERRY KURVE COMMODOE CHAIR

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your **TIMBERRY KURVE COMMODOE CHAIR** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

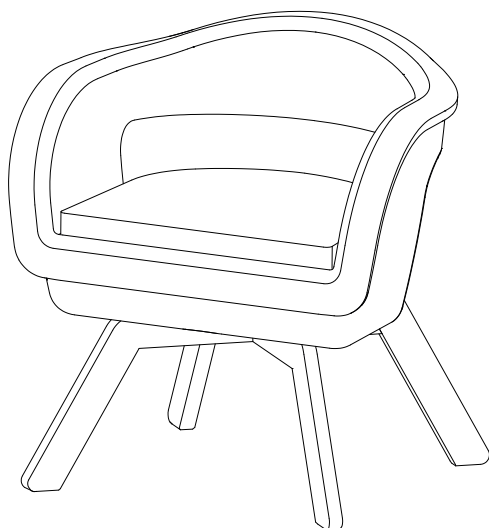A

B

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

MD

IFU-REF 56551/0625-1

REF 55500 + 56551 TO 56560





VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)

@EllisHealthcare

❶ ASSEMBLY INSTRUCTIONS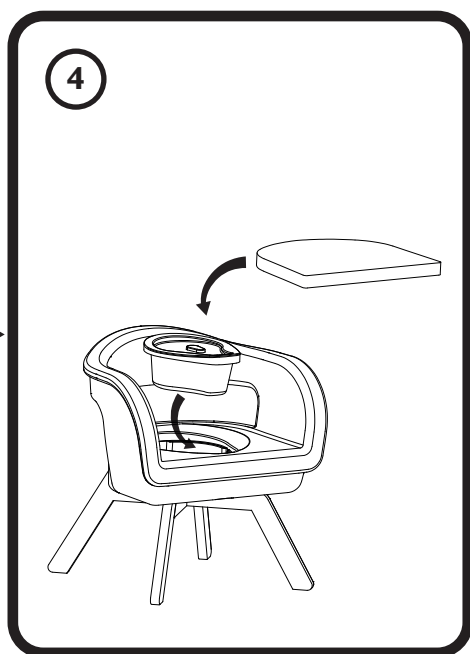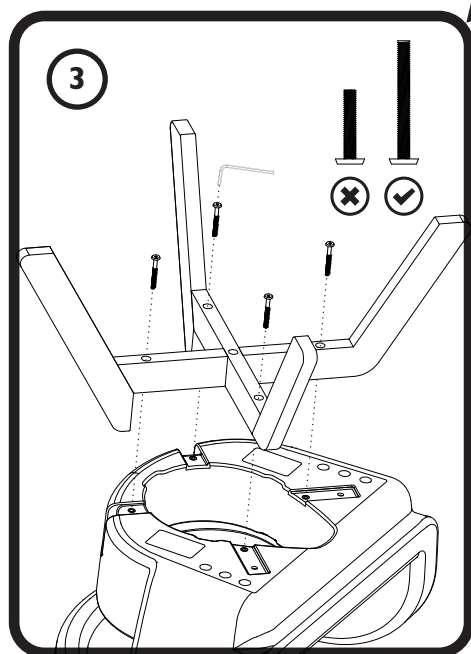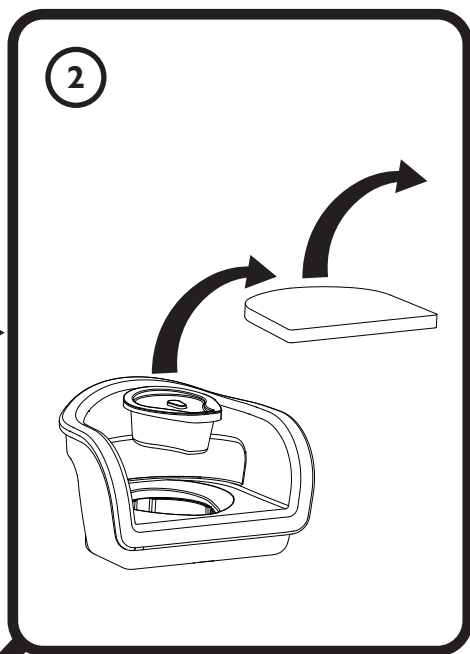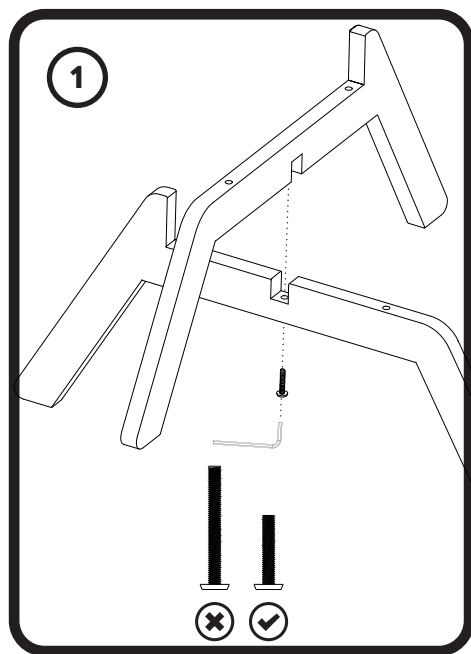


i COMMODE INSTRUCTIONS



1



2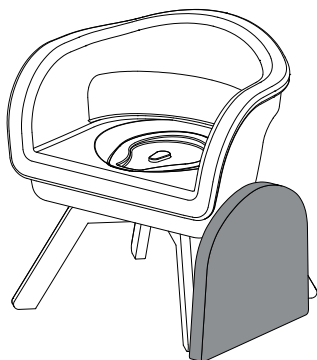

3

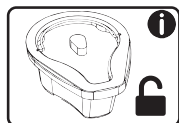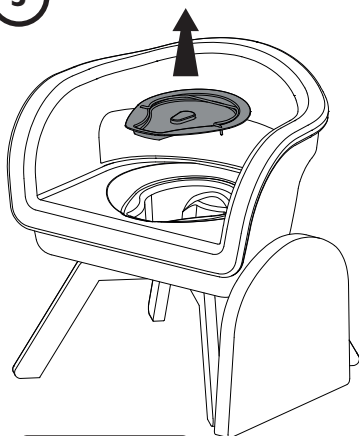4

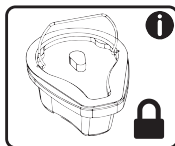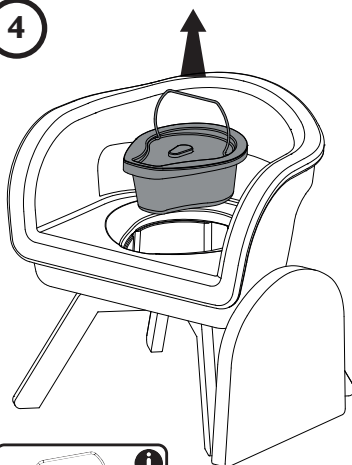
ASSEMBLY INSTRUCTIONS: 1. SLOT THE LEGS TOGETHER AND SECURE USING THE SMALL SCREW. 2. REMOVE THE SEAT PAD AND PAN. 3. TURN THE SEAT OVER AND SECURE THE LEGS AT THE FOUR FIXING POINTS. 4. RETURN THE PAN AND SEAT PAD. ENSURE PAN HANDLE IS VERTICAL TO LOCK THE LID BEFORE REMOVAL. **COMMODE INSTRUCTIONS:** 1. REMOVE THE SEAT PAD. 2. STORE THE SEAT PAD IN A SAFE LOCATION WHILE USING AS A COMMODE. 3. REMOVE THE LID FROM THE PAN - ENSURE THE HANDLE REMAINS DOWN TO ALLOW REMOVAL. USE THE COMMODE. 4. RETURN THE LID. LIFT THE PAN FROM THE COMMODE ENSURING THE HANDLE IS VERTICAL TO LOCK THE LID. FOLLOWING WASTE DISPOSAL, RETURN THE PAN AND SEAT PAD.

CHECK FOR STABILITY BEFORE EACH USE – STATIC USE ONLY. CARE SHOULD BE TAKEN TO SIT IN A CONTROLLED MANNER AND AVOID SUDDEN OR FORCEFUL IMPACT TO THE SEAT AREA. IT IS THE RESPONSIBILITY OF THE OWNER TO CARRY OUT AN INITIAL AND REGULAR RISK ASSESSMENTS. PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS AND ADJUST AS NECESSARY.

A = MAXIMUM USER WEIGHT.
B = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS)

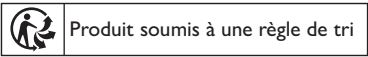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